



USER GUIDE



FLS CONVERTER BUNDLE

50370

LEARN MORE AT

fls-products.com

Limbs & Things Inc. | P.O. Box 15669 | Savannah, GA 31416, USA

PACKAGE
SUPPLIED

Converter



Cable Splitter



Additional Video Cable



SAFETY &
CLEANING
INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE
USING YOUR FLS CONVERTER BUNDLE



Follow all instructions.
Keep these instructions.
Heed all warnings.



Do not install near any heat sources,
do not expose the product to flames
or use near naked flame sources.
Do not use flammable solvents near or
on this product.



Only use attachments/accessories
specified by Limbs & Things




Do not use this apparatus near water,
this product contains electronics.

TAKE A LOOK
AT OUR **VIDEO**


VISIT THE WEBSITE
fls-products.com

OR YOUTUBE CHANNEL
youtube.com/limbsandthings

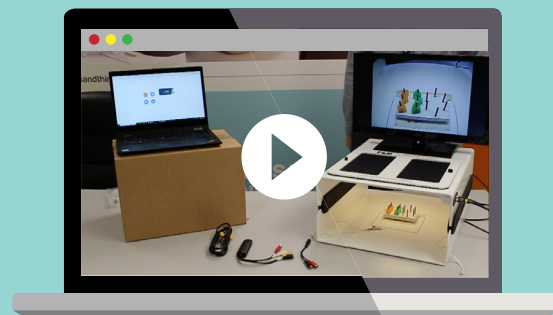

WIRING SET UP

- 1 Set up the FLS box trainer making sure all components are switched on

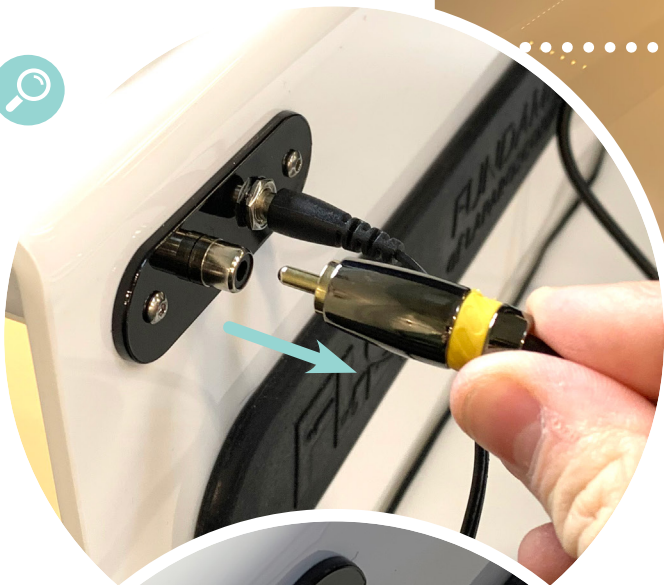

- 2 Unplug the connecting (yellow) video cable from the FLS box



- 3 Plug in the newly supplied cable splitter into the receiving port where you have just disconnected the yellow video cable


SET UP


- 4 Re-plug the video cable back into one of the two ports on the cable splitter

 Video feed should reappear




- 5 Plug the Video Converter USB port into your laptop / PC


a. A constant green light should show
b. A notification will appear on your device telling you automatic download has begun

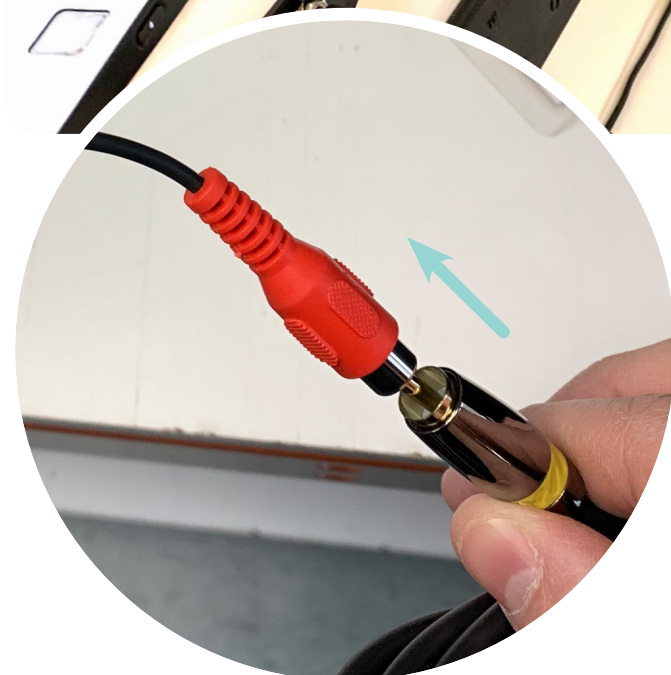

- 6 Take second supplied yellow video cable and attach one end into the other available port on the cable splitter

SET UP

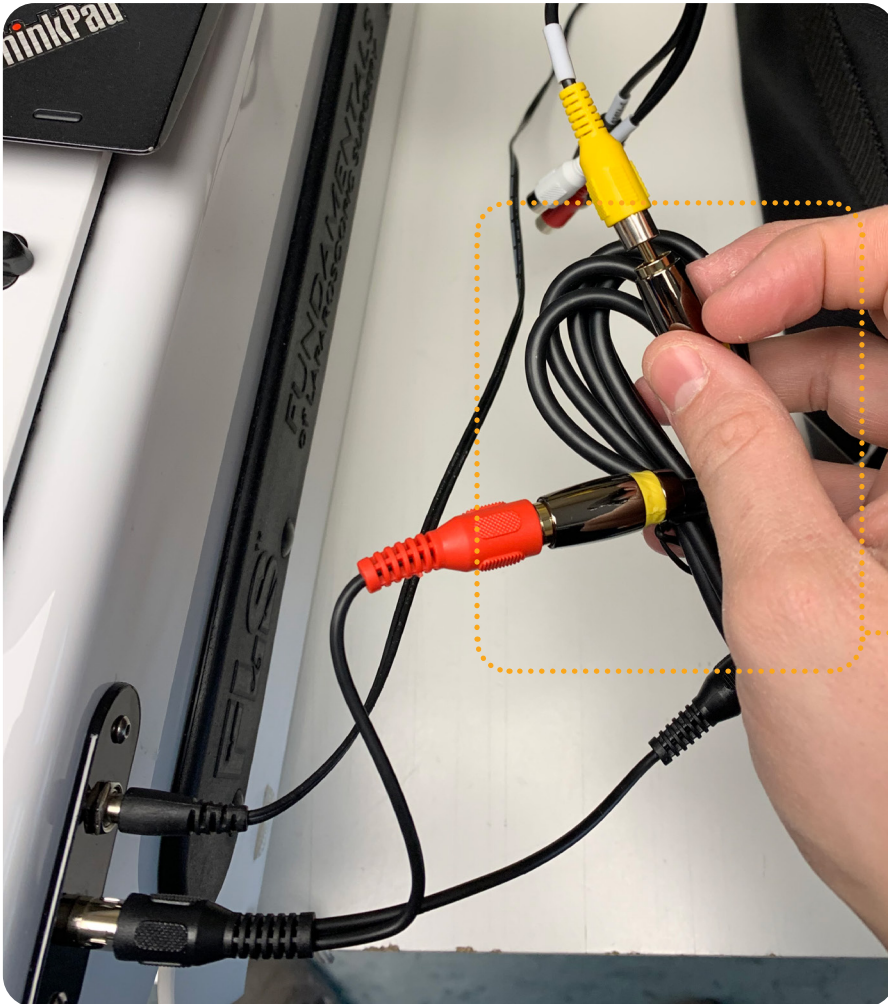

7 Take the second end and attach to the yellow port on the video Converter dongle




This is the additional video cable



Wiring is now set up


SHARE
SCREEN

Click the Zoom Meeting link
provided by FLS

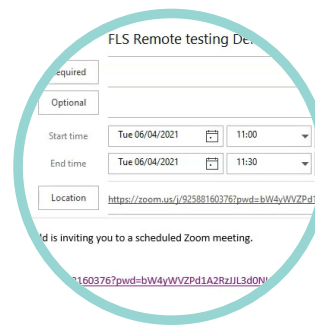


1

Either open the zoom meeting with the app or continue using the website

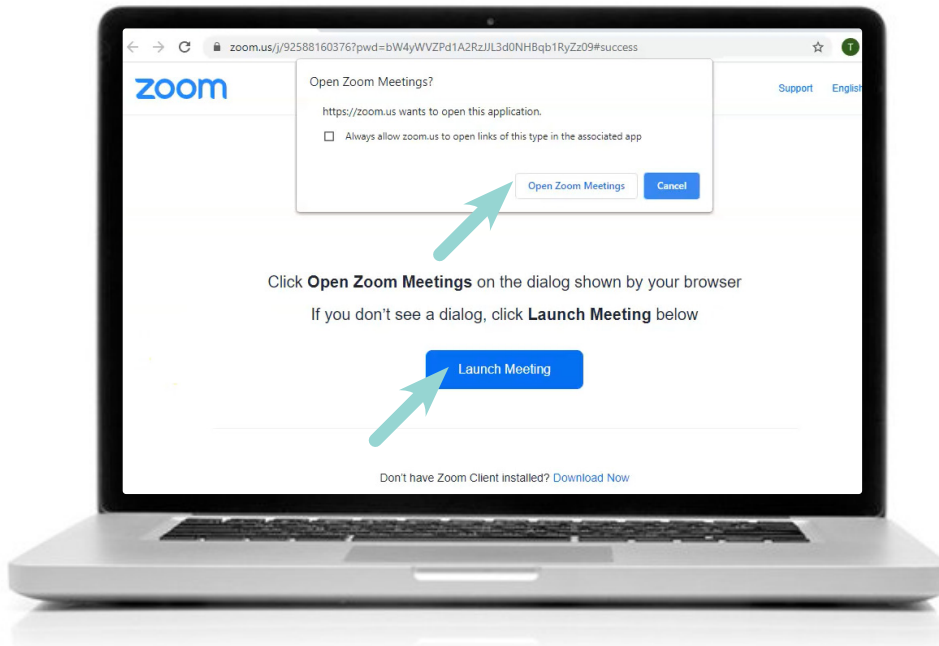


2

Start a Zoom call

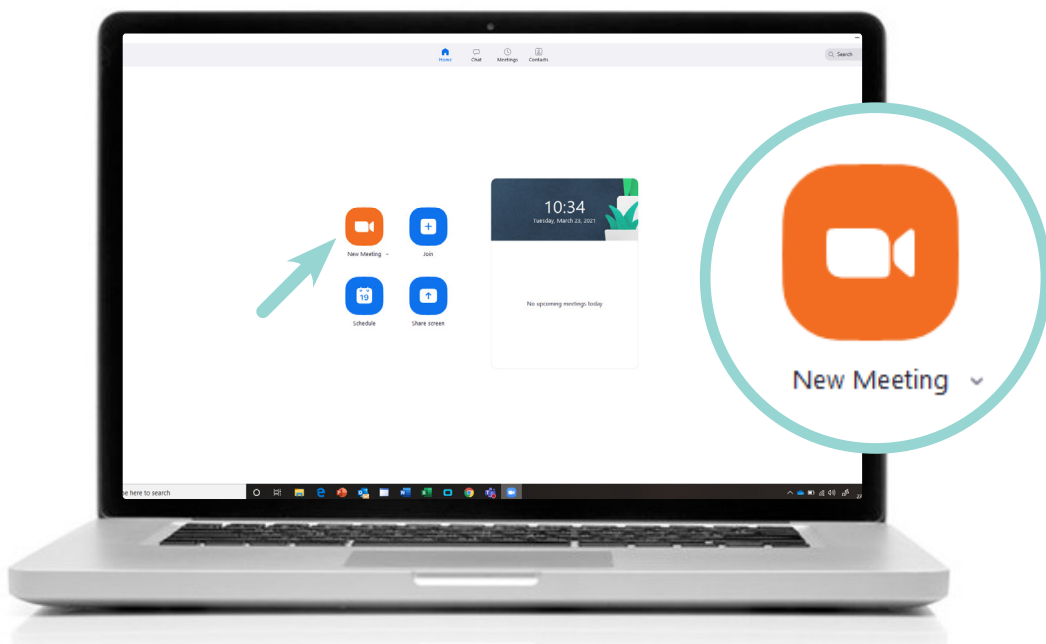


SHARE SCREEN

3 Click 'Share Screen'

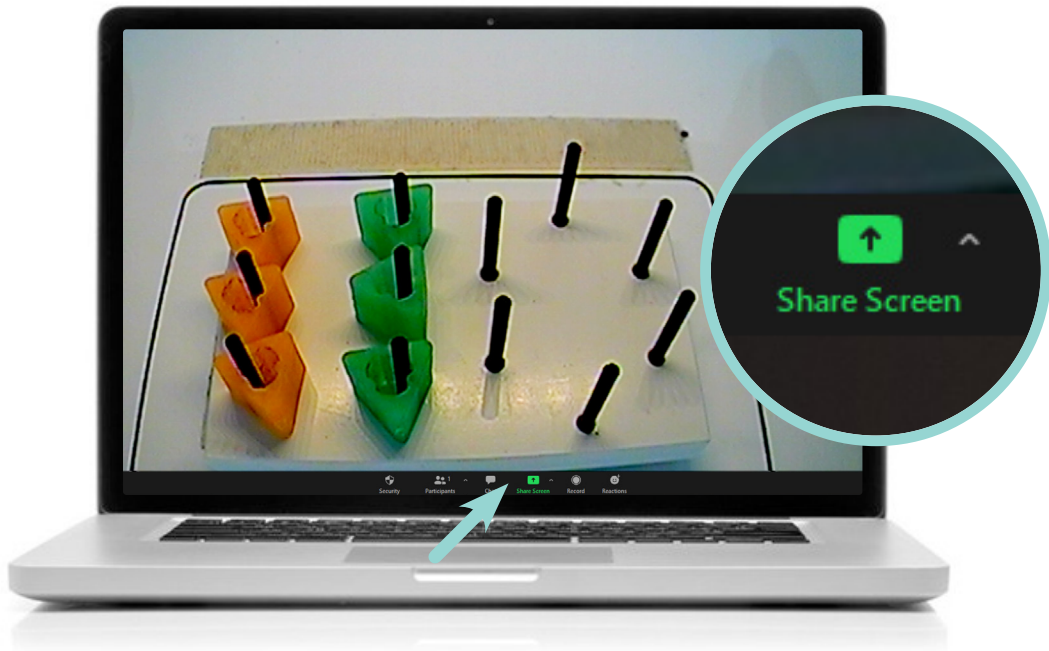


4 Following options will show – click 'Advanced' tab

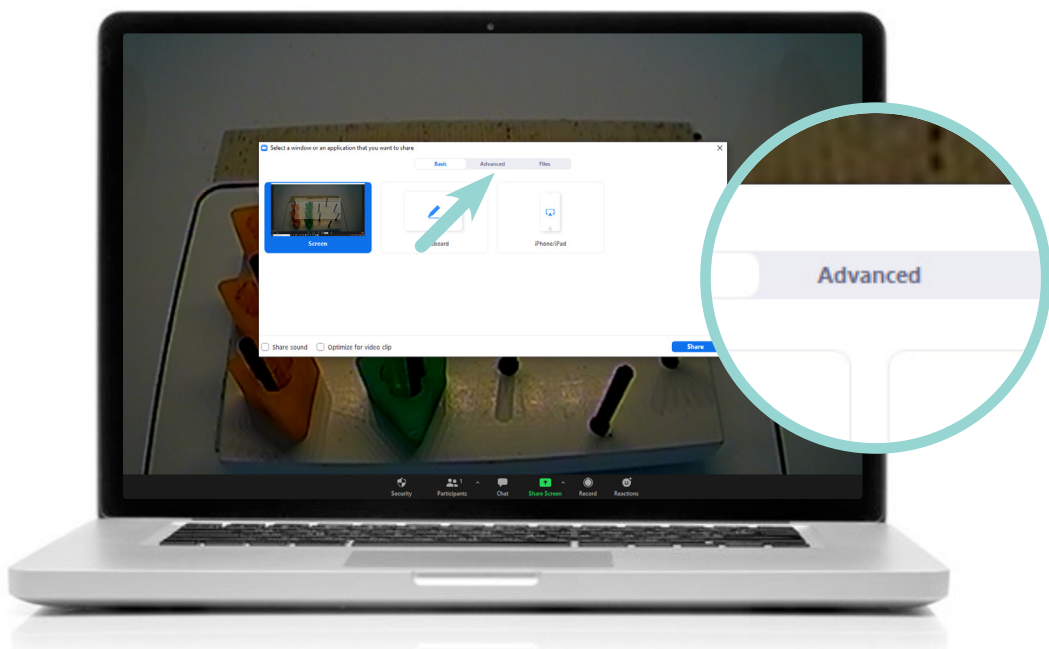

5

In Advanced options select the 'Content from 2nd Camera' option. Press 'Share' in bottom right.

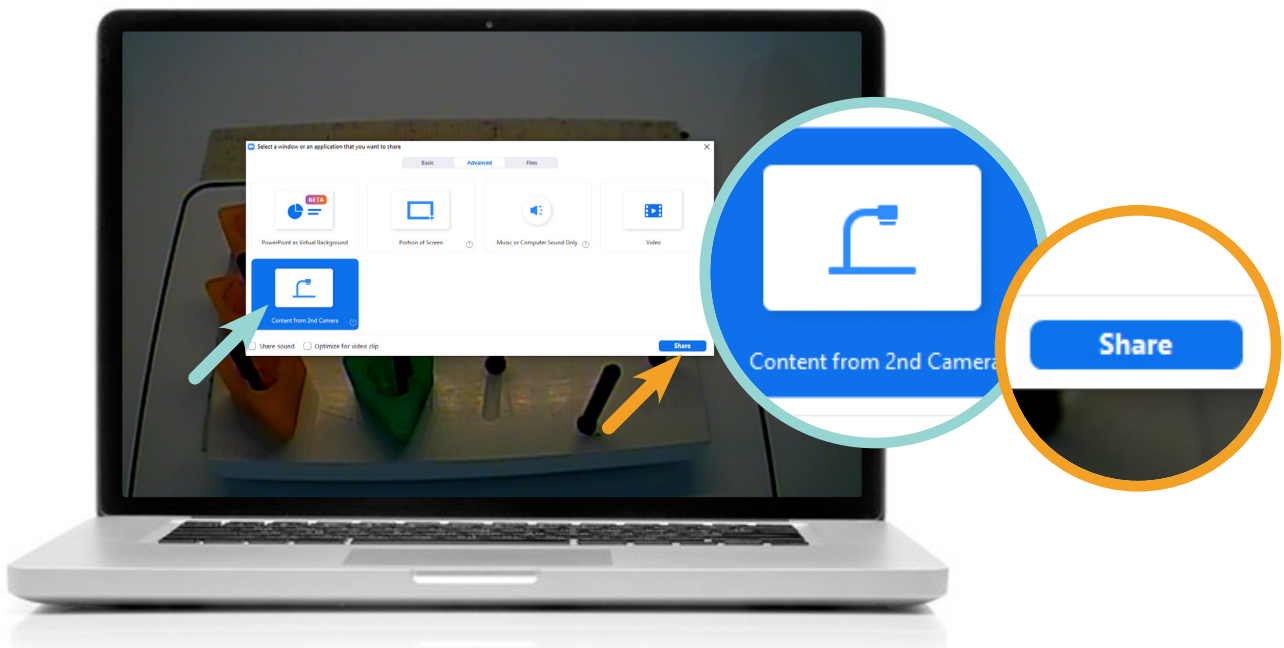

6

Click 'Switch Camera' to show the video feed from the FLS box

