

## User Guide

# PROMPT FLEX COMPRESSION SUTURE UTERUS

80111



For more skills training products visit

limbsandthings.com

Limbs & Things Ltd.
Sussex Street, St Philips,
Bristol, BS2 ORA, UK
E: sales@limbsandthings.com
T: +44 (0) 117 311 0500

# PROMPT FLEX COMPRESSION SUTURE UTERUS

### 80111

An additional module for the **PROMPT Flex Standard** or **Advanced** birthing simulator for practicing all common types of Compression Suture (including B-Lynch, Hayman and Pereira) and ligating the uterine artery.

Designed to be used with our **Pre-Incised C-Section Skin**, it can also be used with our mock blood to simulate vaginal bleeding.

Can also be used with the **PPH Module (80101)** to simulate bleeding.



# Safety instructions

This product is not a toy, and should only be used as instructed in this User Guide.

When using sharp instruments such as scalpels and needles be sure to take care and dispose of correctly.

The Compression Suture Uterus can only be used with the Limbs & Things PROMPT Flex Simulators.

## Skills

- All common types of Compression Suture procedures (including B-Lynch, Hayman and Pereira)
- Dissection of broad ligament
- Ligation of uterine arteries

## Product Qualities

### **ANATOMY**

- A representation of post-partum uterus including uterine arteries and broad ligament

#### **KEY FEATURES**

- Uterine arteries can be fluid filled
- Ligation of arteries is possible
- Multiple uses when conducting the compression suture


### **VERSATILITY**

- Can be quickly and easily fitted into the PROMPT Flex birthing mother



• Compression Suture Uterus (x2)

## SETTING UP THE PRODUCT

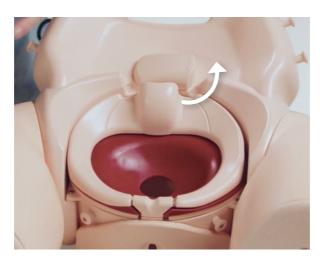

Rotate the securing bar 90 degrees and remove the ring clamp from the PROMPT Flex Lay in the cervix, if it is not already in place



Ensure the elastic hoops are sat behind the vessels





To attach the Uterus, feed the white hoops through the ring clamp





With the uterus attached, align the front of the ring clamp





Then turn the securing bar clockwise to lock in place





Squeeze the end of the uterus






Push into cervix opening for a snug fit





If you already have the PPH Module (80101), it can be attached to the Uterus module to simulate bleeding. This can be done by attaching the plastic connector on the Uterus to the plastic connector on the PPH Module.

Refer to the 80101 user guide for detailed product use

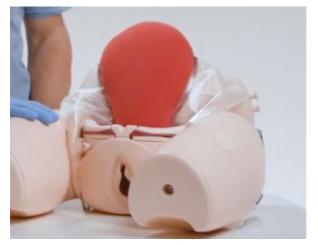



The Uterus is now in place





For heightened realism, lay the pre-incised delivery skin over the birthing mother






The product is now ready to use

You may also be interested in...

# PROMPT FLEX - PPH MODULE

80101

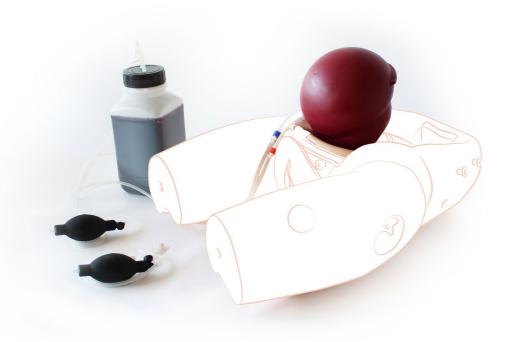

To find out more visit:

limbsandthings.com



Part No: 165-018 Issue 1, DEC 2017 © 2017 Limbs & Things

