



## USER GUIDE



# ADVANCED EPIDURAL & LUMBAR PUNCTURE MODEL

This product is available  
in light and dark skin tones

**61001**

**61024**

FOR MORE SKILLS TRAINING PRODUCTS VISIT

[limbsandthings.com](https://limbsandthings.com)

Sussex Street, St Philips, Bristol, BS2 0RA, UK | [sales@limbsandthings.com](mailto:sales@limbsandthings.com) | +44 (0) 117 311 0500

## INDEX

PACKAGE SUPPLIED	3
SAFETY & CLEANING INSTRUCTIONS	4
SET UP	5
• STAND	5
• CARTRIDGE	7
• SKIN PAD & FAT PADS	9
• DURA	10
CARE & MAINTENANCE	11

TAKE A LOOK  
AT OUR **VIDEO**



VISIT THE WEBSITE  
[limbsandthings.com](https://limbsandthings.com)

OR YOUTUBE CHANNEL  
[youtube.com/limbsandthings](https://youtube.com/limbsandthings)

PACKAGE  
SUPPLIED

Epidural & Lumbar Puncture Stand



Epidural & Lumbar Puncture Water Container

61022



Epidural & Lumbar Puncture Torso with Torso Skin



Lumbar Puncture Insert - Bone

61010



Lumbar Puncture Insert - Tissue (x2)

61011



Epidural Insert (x2)

61012



Advanced Epidural Insert (x2)


61013



Epidural & Lumbar Puncture Skin Pad


61015

61027

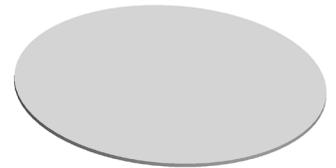

Epidural & Lumbar Puncture Fat Pad (x3)

61016

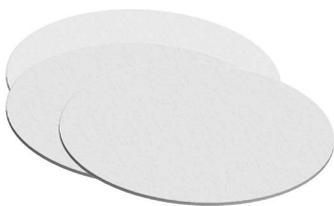

Epidural & Lumbar Puncture Standard Dura (x2)

61017



Epidural & Lumbar Puncture Advanced Dura (x4)

61018



Epidural & Lumbar Puncture Insert Cartridge


61020



Epidural & Lumbar Puncture Sacral Insert

61021

61028

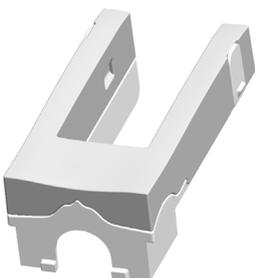

ART mat

61030



YOU WILL NEED (NOT SUPPLIED)

Spinal Needles



Manometer



Epidural Kit



## SAFETY & CLEANING INSTRUCTIONS

## READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR ADVANCED EPIDURAL & LUMBAR PUNCTURE TRAINER

### DO's



Follow all instructions. Keep these instructions. Heed all warnings.

Only use attachments/accessories specified by Limbs & Things.



Refer all servicing to qualified service personnel. Servicing is required when the apparatus has been damaged.

Drain all water from the product and clean with a damp soft cloth or sponge, using only warm water with mild detergent.



### DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.



Do not allow young children to use this product.

### STORAGE & TRANSPORTATION





Always ensure that the products are properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.



SET UP

STAND


1

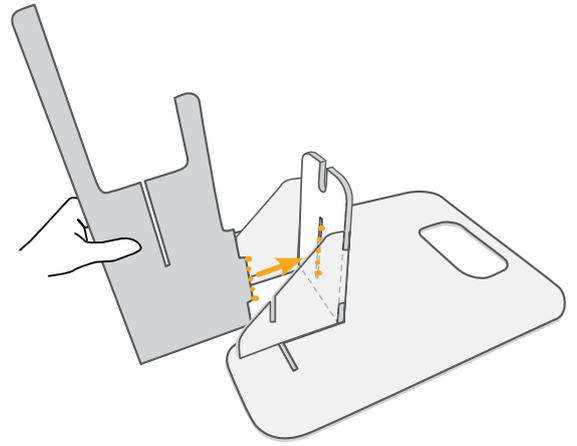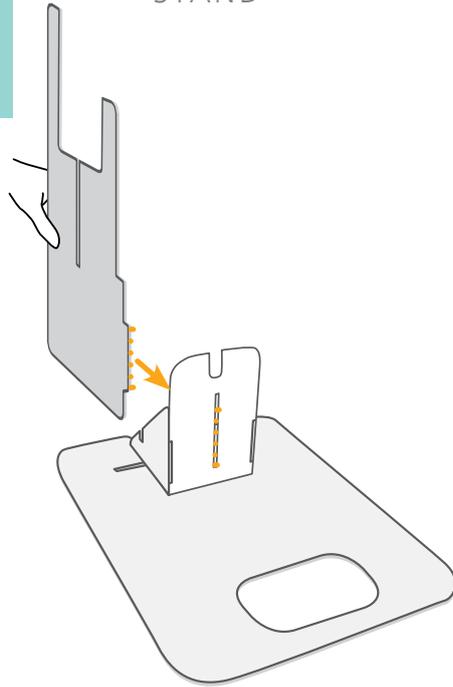


2

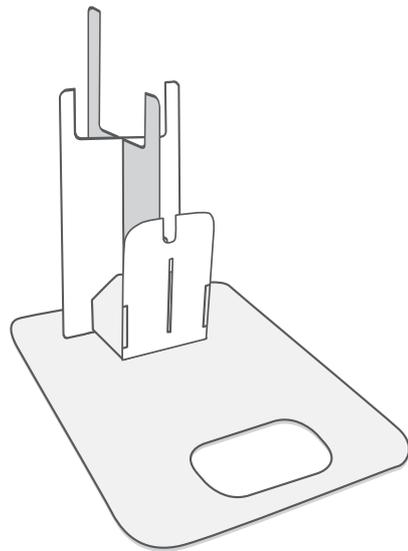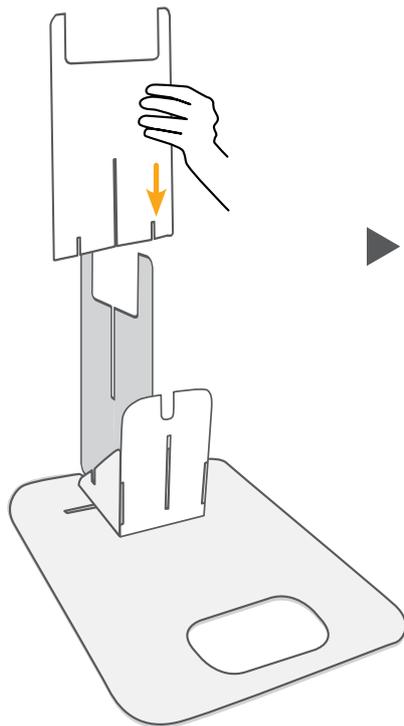

3

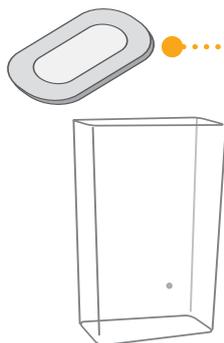


Use lid as drip tray

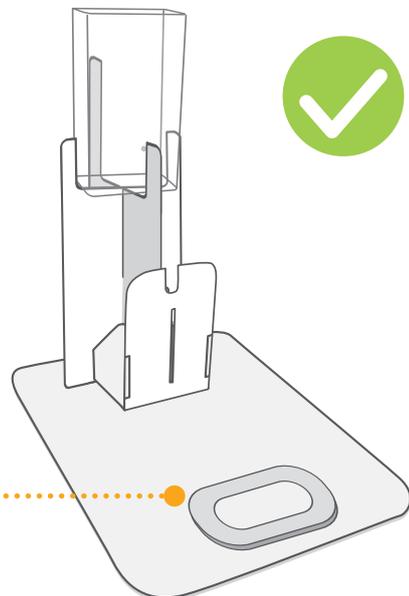


SET UP

STAND

4

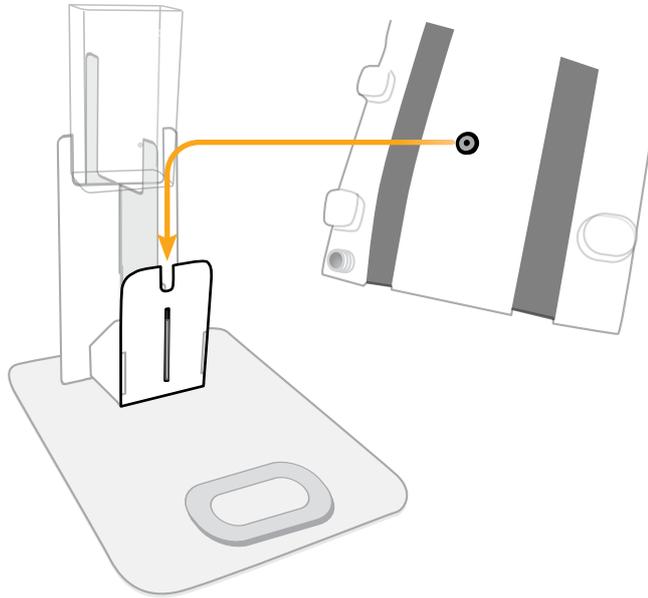


Line up back of torso with the stand and slot down

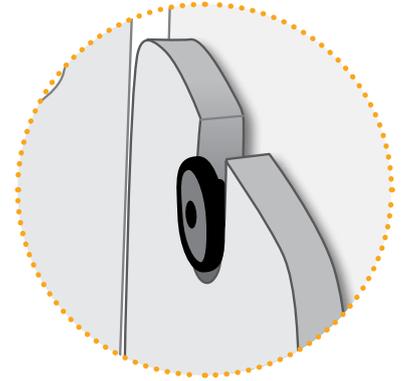

5



Torso can be upright or left lateral

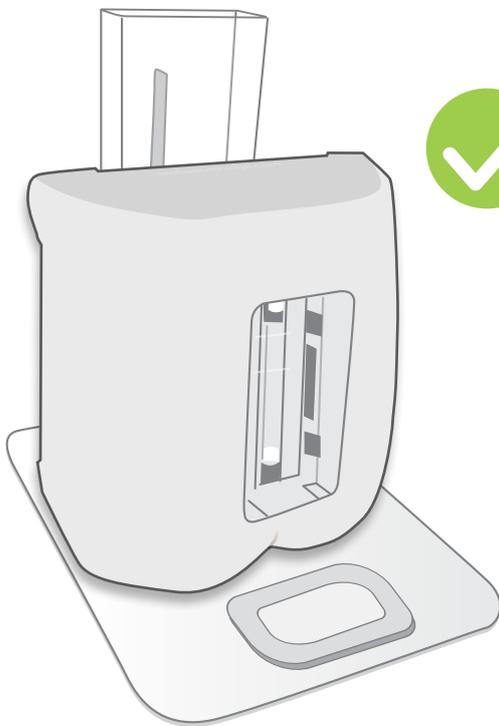

SET UP

CARTRIDGE

1

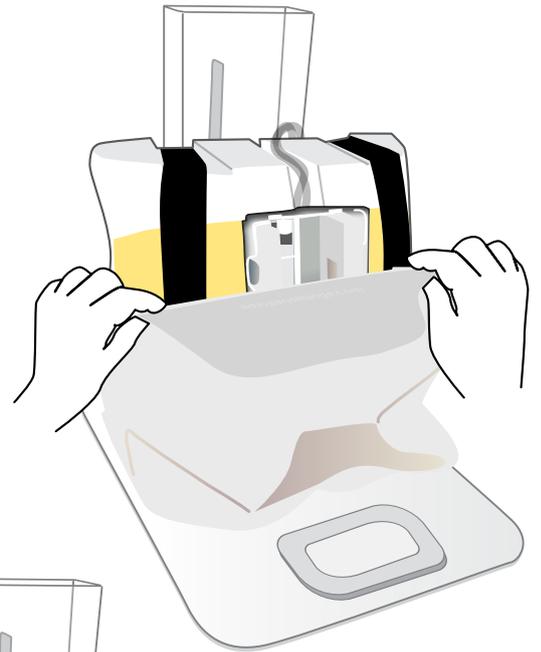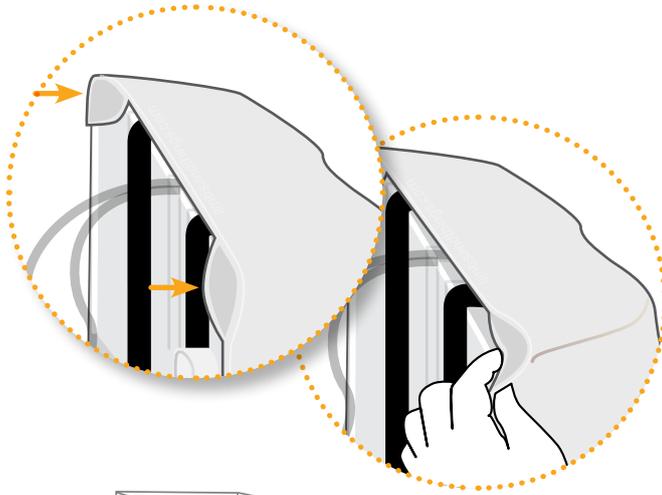

2



3



Choose a Standard Dura  
for durability



or an Advanced Dura  
for more realistic training




SET UP

CARTRIDGE

4

Select insert:


Lumbar Puncture

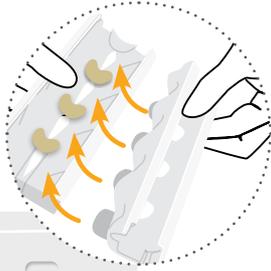

Advanced Epidural



5

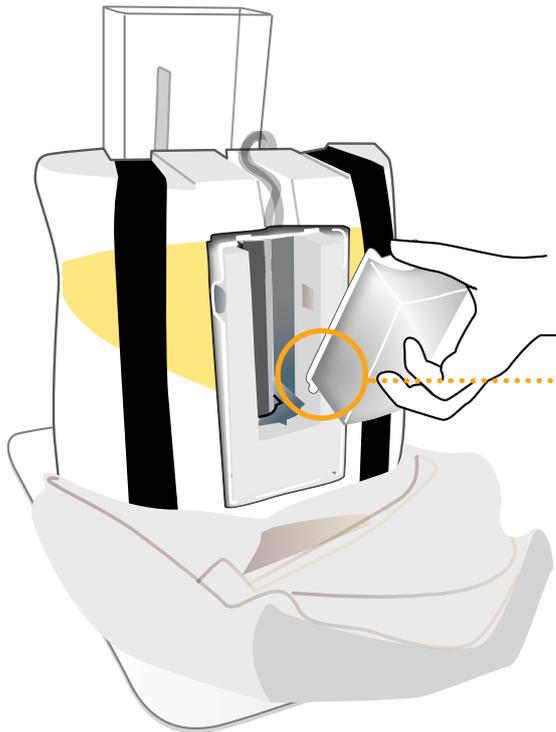

Fit lower end of insert (with teeth) first





## SET UP

## SKIN PAD & FAT PADS



Add one to three fat pads to create a more obese patient, or use without fat pads

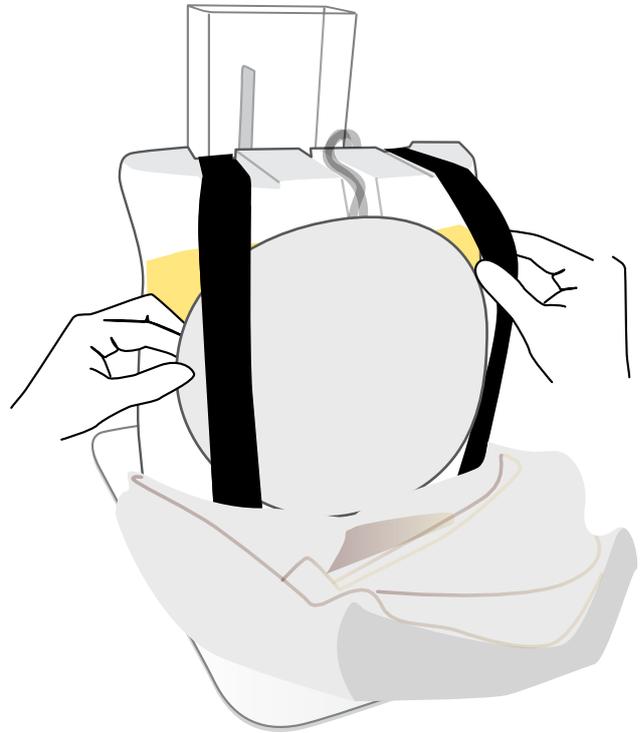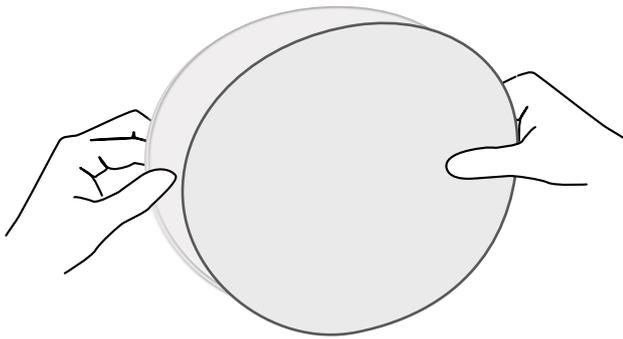



Rotating the skin provides greater usage, and prevents previous puncture marks locating the injection site




SET UP

DURA


1

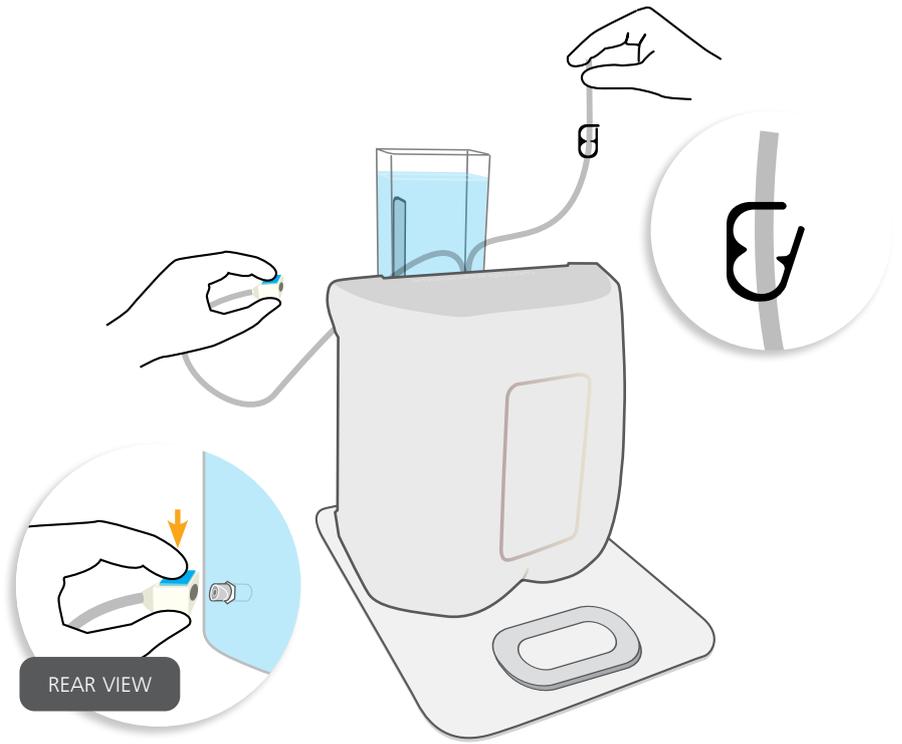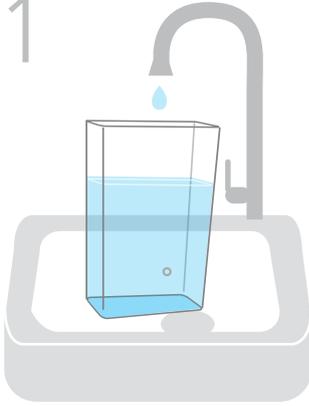

2

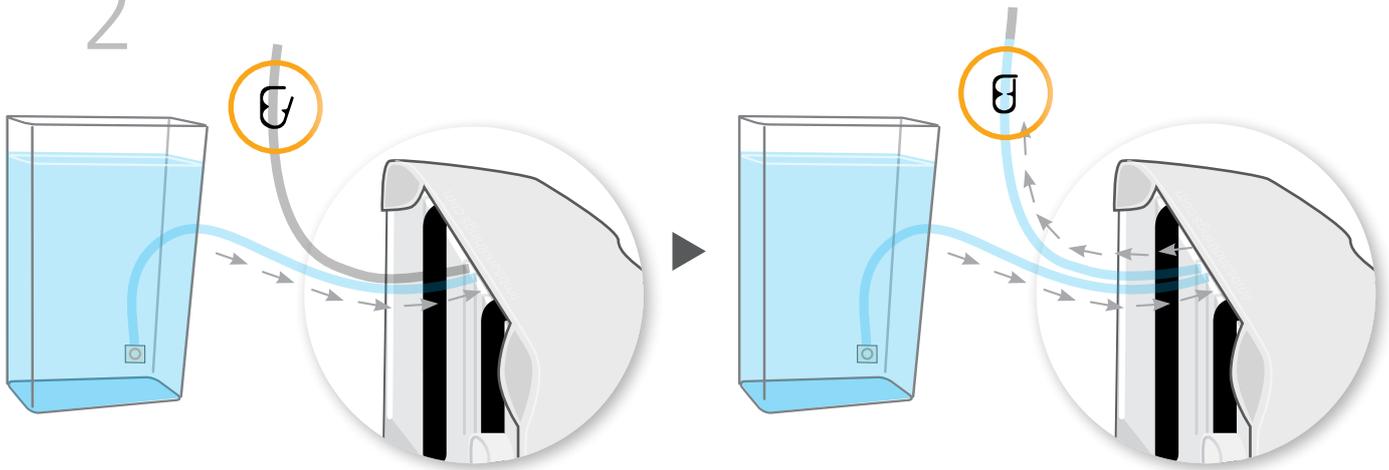


3

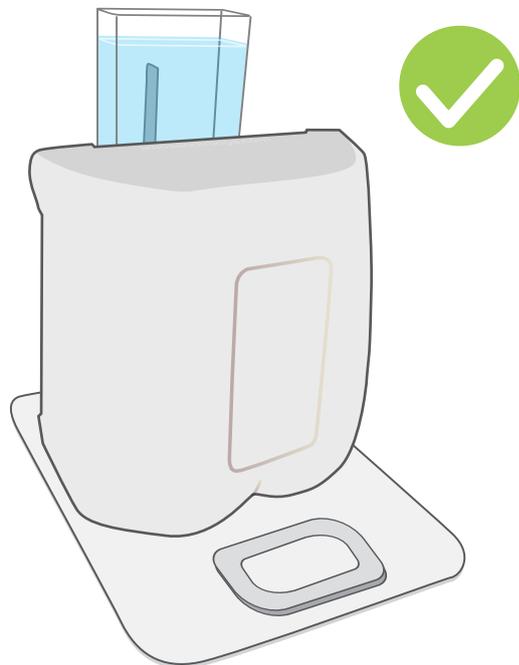


# CARE & MAINTENANCE

1

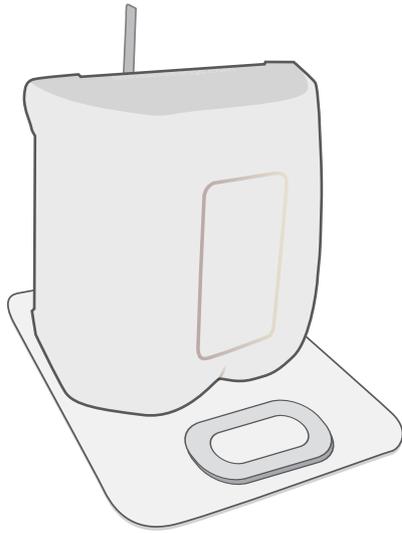





2

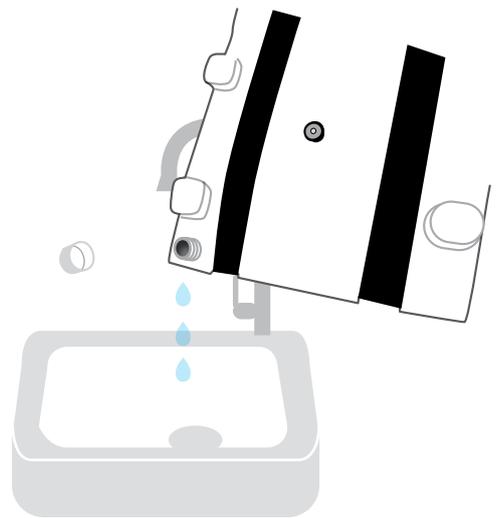

3



Dry all the parts with paper towel

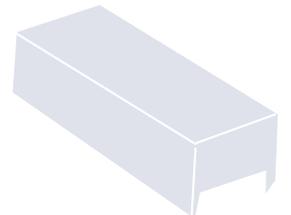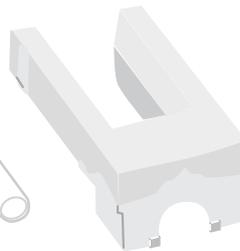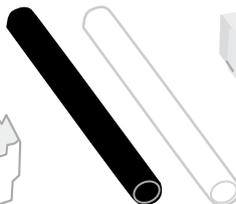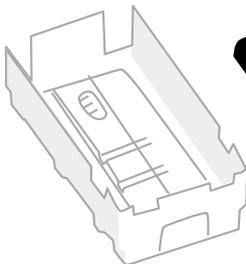

Refit all the parts when dry

YOU MAY  
ALSO BE  
INTERESTED IN...

ULTRASOUND  
EPIDURAL  
& LUMBAR  
PUNCTURE MODEL

61002 61025



PARACENTESIS  
TRAINER

60100 60111

