



USER GUIDE



BREAST EXAMINATION TRAINER

ADVANCED



This product is available
in light and dark skin tones

40201

41201

FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500

INDEX

PACKAGE SUPPLIED	2
SAFETY & CLEANING INSTRUCTIONS	3
SET UP	4
• SETTING UP PATHOLOGIES	4
• REMOVAL & REFITTING BREAST INSERTS	6
• SETTING UP THE PADS	7
TRAINING	11
• FITTING THE BREAST SKIN FOR WEARABLE USE	11
CLEANING & STORAGE	12
• CLEANING & STORAGE AFTER USE	12

PACKAGE SUPPLIED

Wearable Examination Breasts



Breast Examination Pathologies

40204



Breast Examination Inserts

40205



Lymph Node Pads

40203

41203



Breast Pathologies Supports

40206



Breast Back Plates

40207



Breast Examination Torso

40208



SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR BREAST EXAMINATION TRAINER

DO's



Follow all instructions. Keep these instructions. Heed all warnings.

To stop the insert from sticking to the inside of the skin pour a small amount of talcum powder into the recess of the breast skin.



Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.

Clean the product with a damp soft cloth or sponge, using only warm water with mild detergent, after every training session.



STORAGE & TRANSPORTATION



Always ensure that the products are properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.

DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not use any attachments other than those supplied by the manufacturer.



Do not wear any jewellery items.

Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.




SET UP

SETTING UP PATHOLOGIES

1




180°

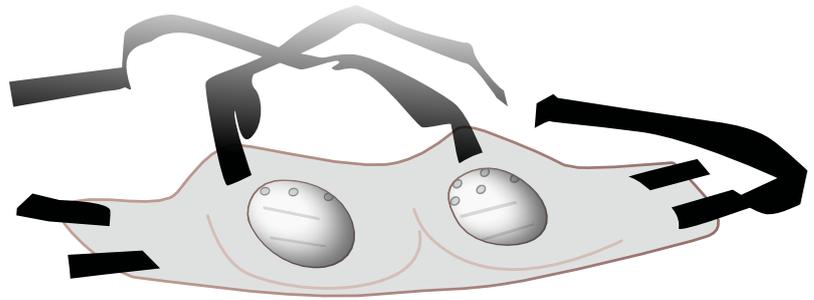

2

Remove back plate and pathology support

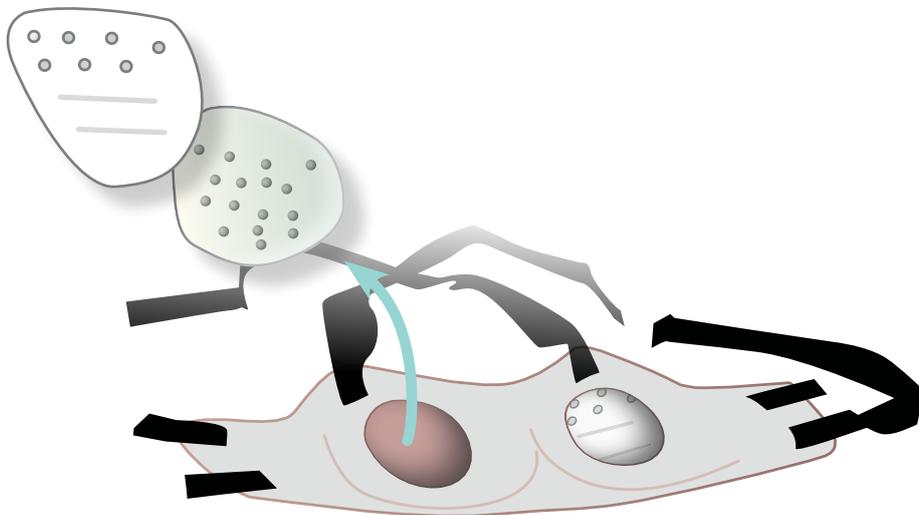


SET UP


SETTING UP PATHOLOGIES




Pathologies can be inserted in both back plate and pathologies support

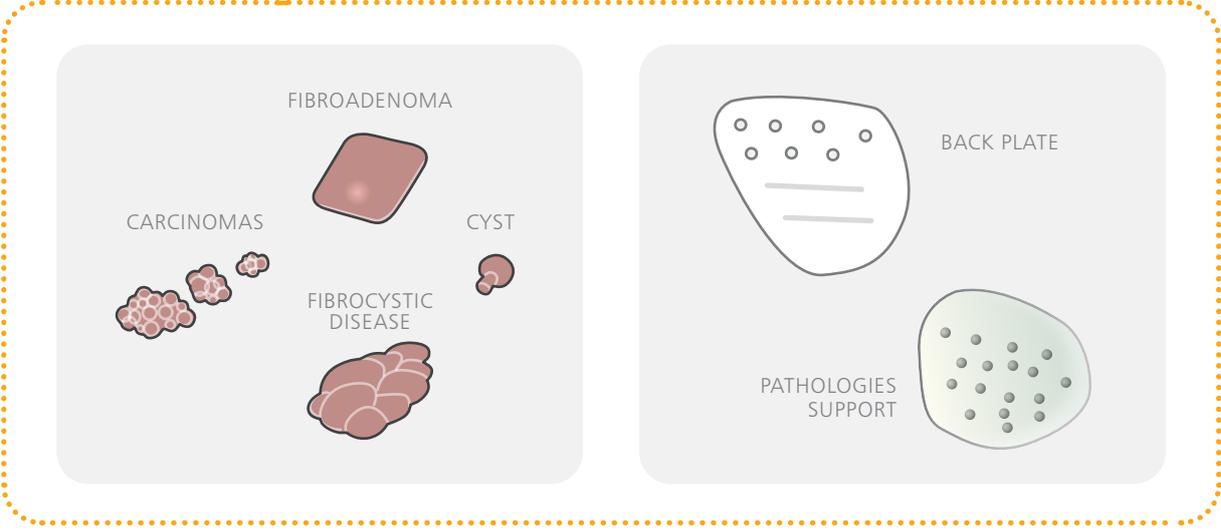


3

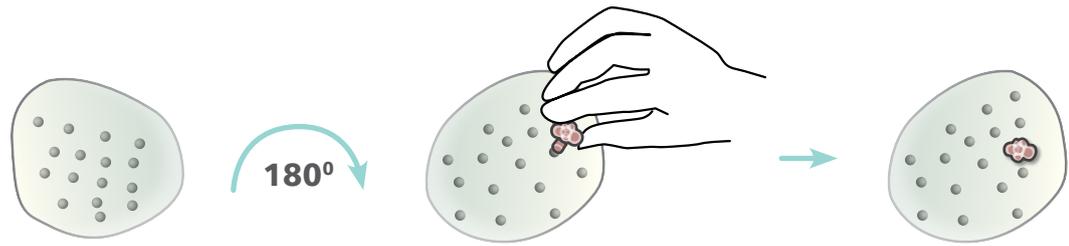

4

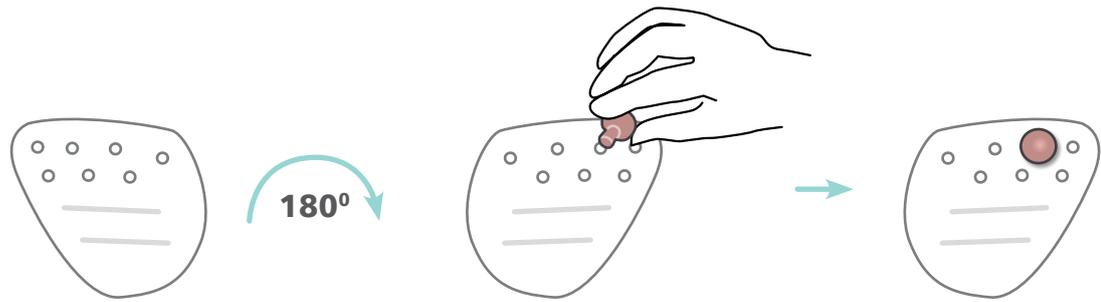


5

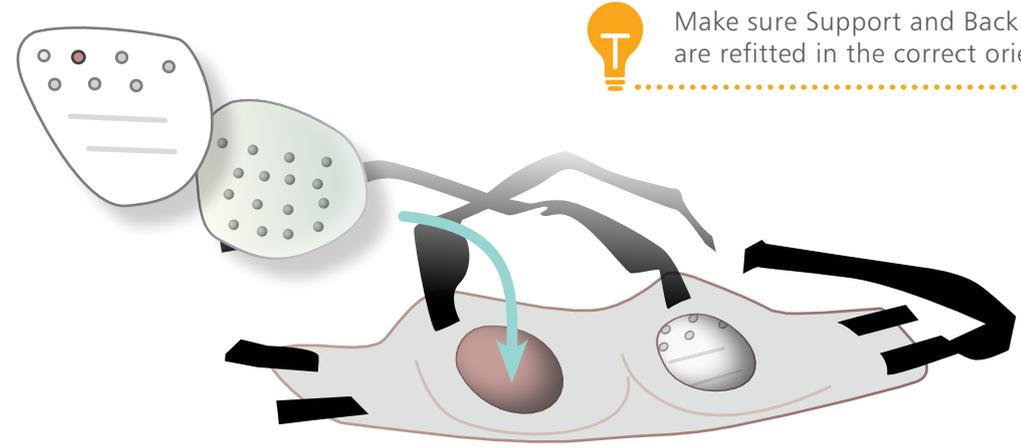


Make sure Support and Back Plate are refitted in the correct orientation

SET UP

REMOVAL & REFITTING BREAST INSERTS

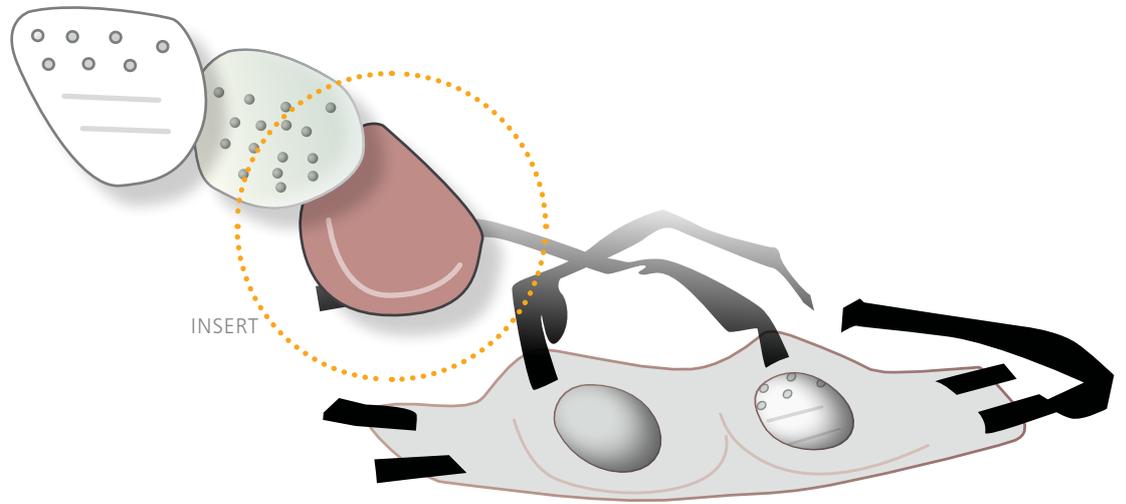


To ensure the breast appearance is smooth, please follow the steps below

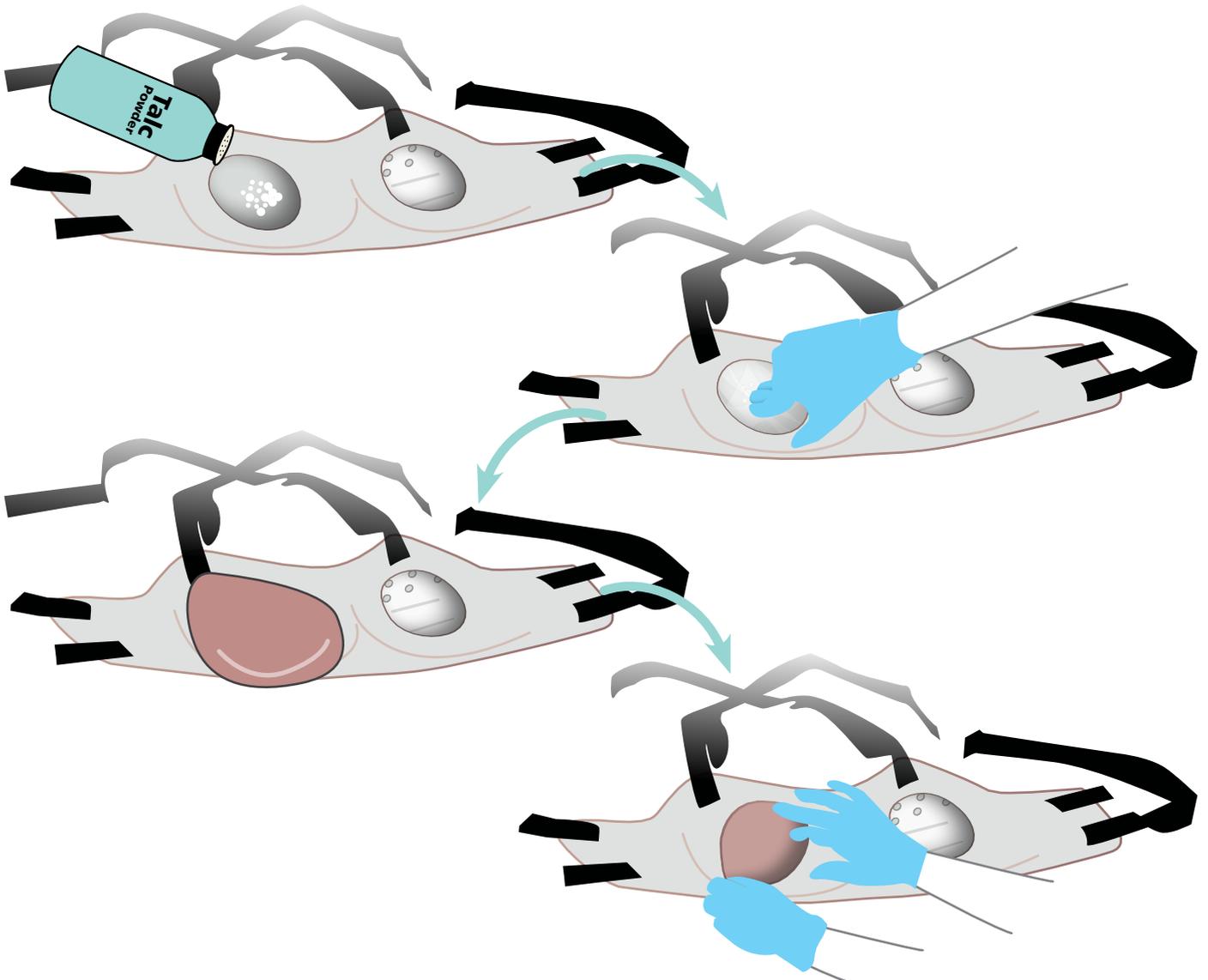

SET UP

SETTING UP THE PADS


Fit the Wearable Examination Breasts onto the Torso

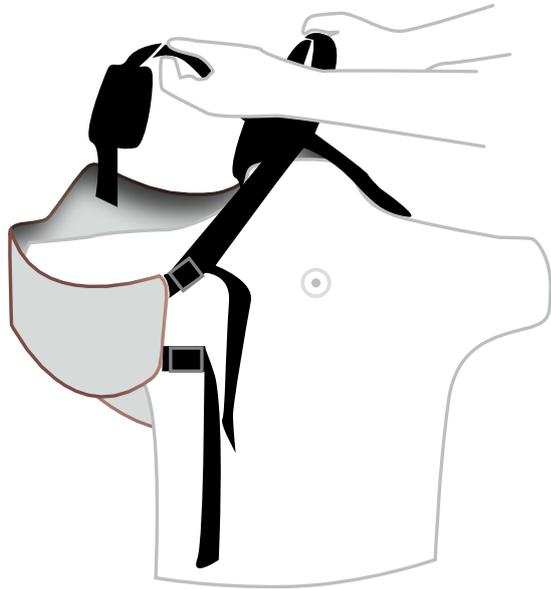

SETTING UP THE AXILLA PADS

Fitting Lymph Nodes in the Axilla Pads


Lay the Axilla Pad down and insert a lymph node into the required position/pocket

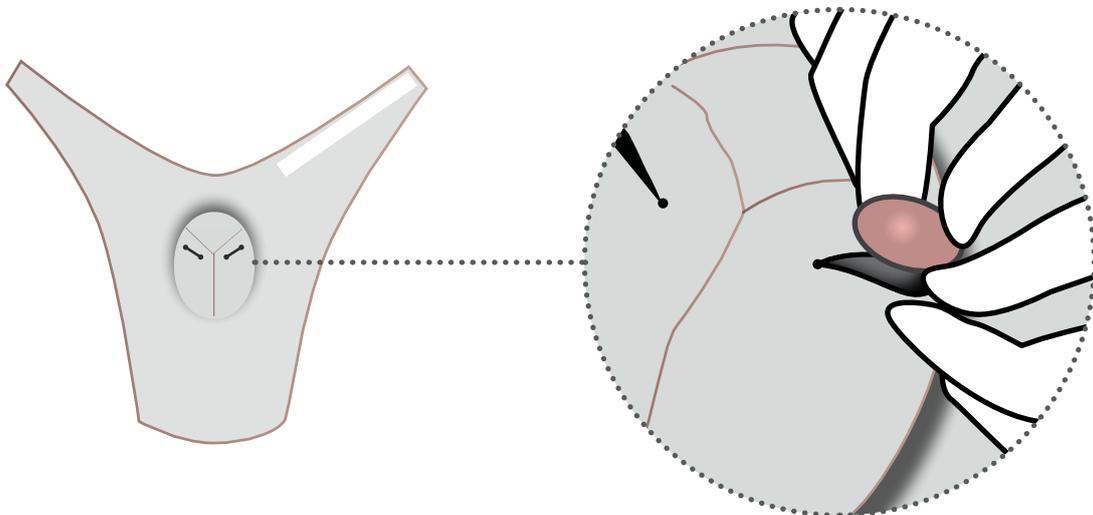

SET UP

SETTING UP THE AXILLA PADS

Fitting & removal of the Axilla Pads

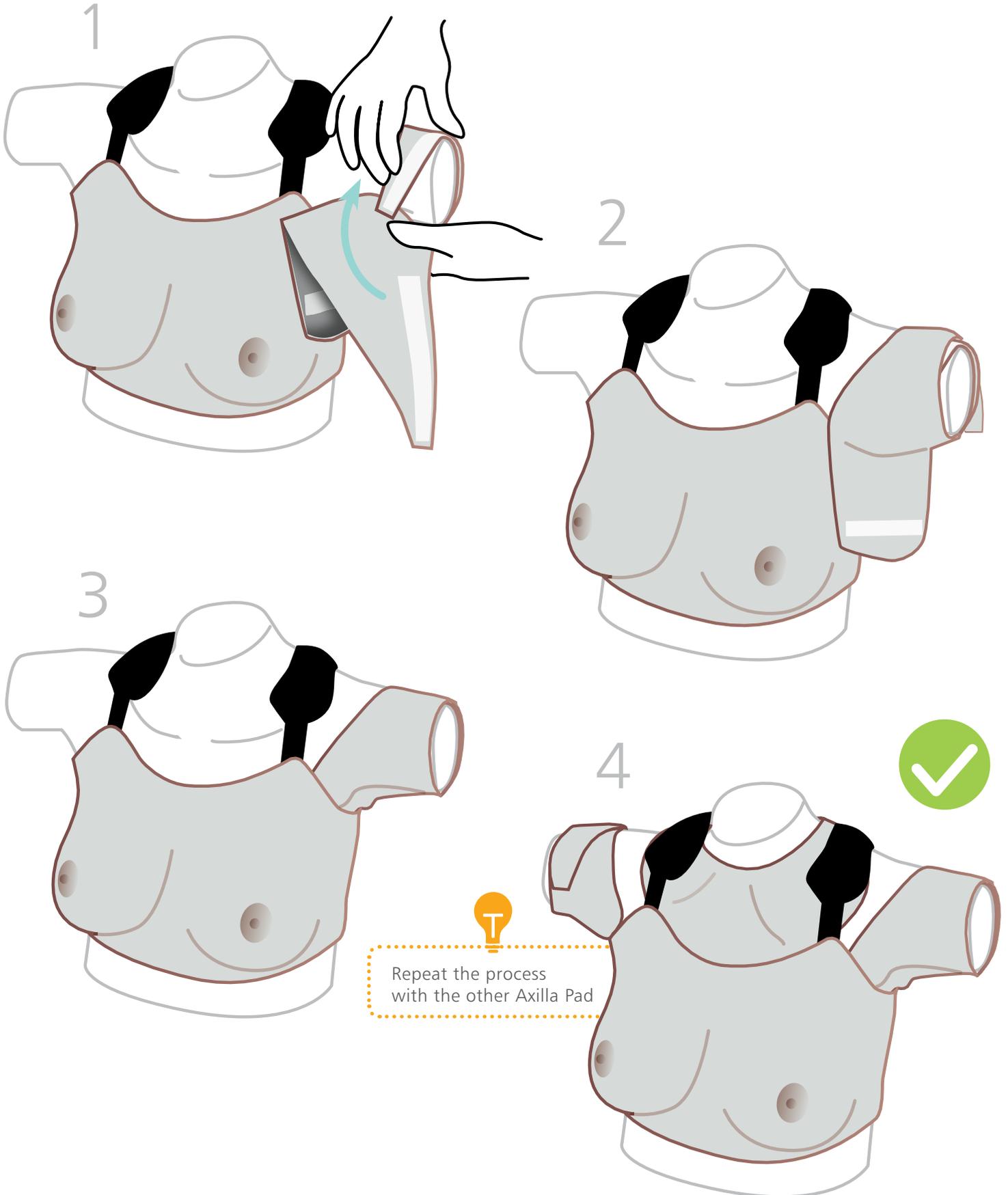


SET UP

SETTING UP THE CLAVICLE PAD

Fitting Lymph Nodes in the Clavicle Pad



Lay the Clavicle Pad down and insert a lymph node into the required position/pocket

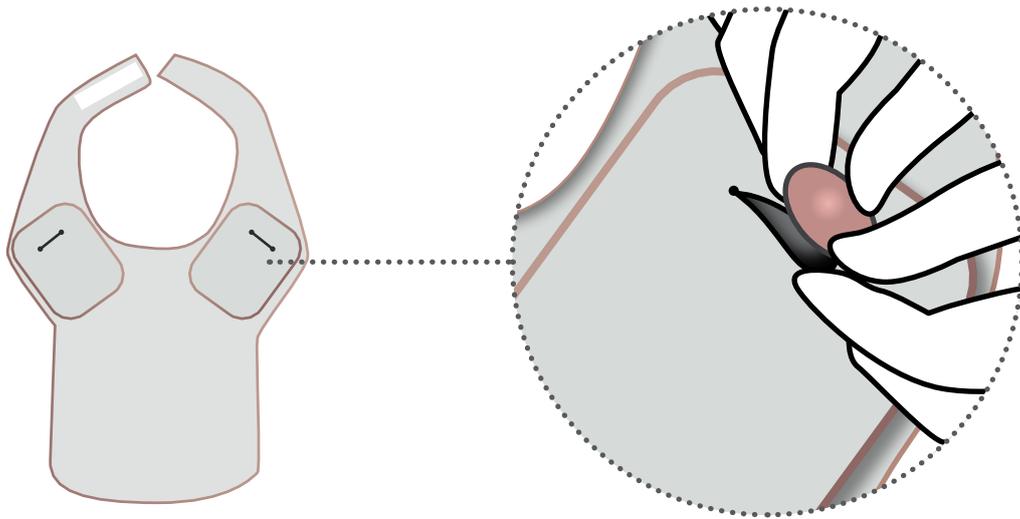

Place the Node in the lower part of the pocket for an infraclavicular node



Place the Node in the upper part of the pocket for a supraclavicular node

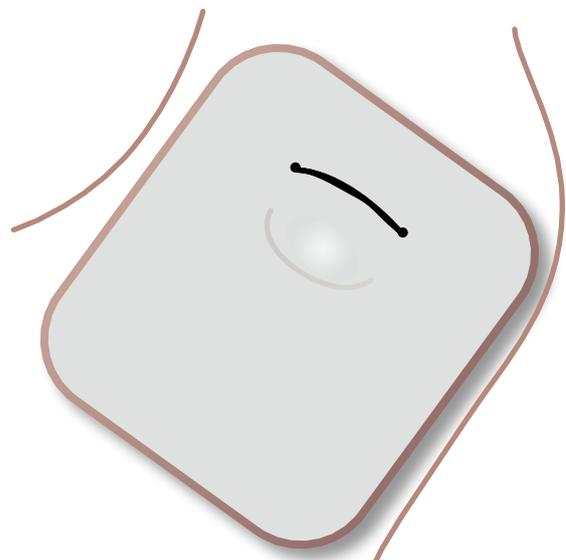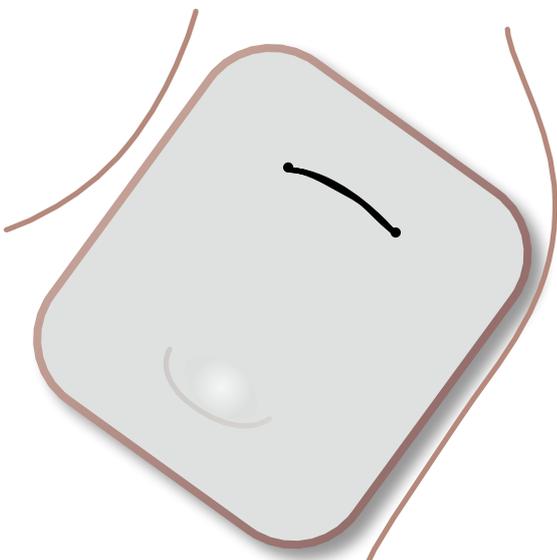


SET UP


SETTING UP THE CLAVICLE PAD

Fitting & removal of the Clavicle Pad


1

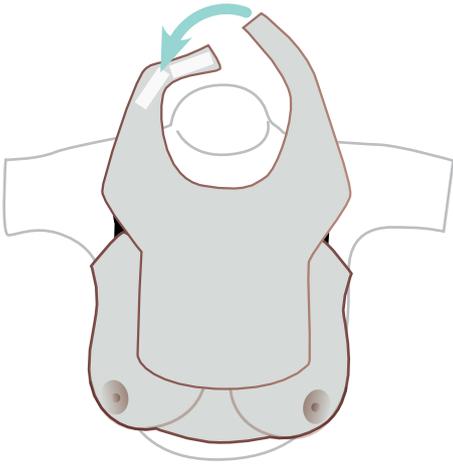

2

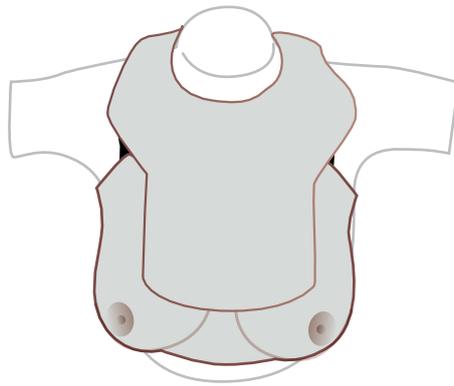

3

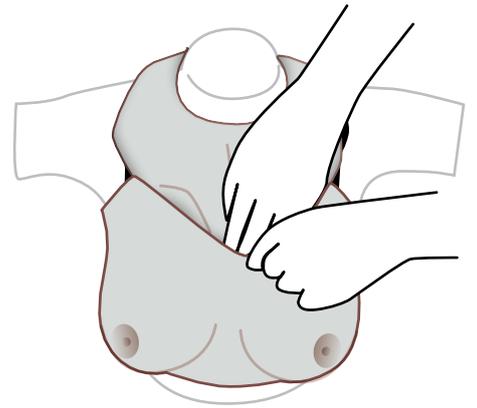

4



TRAINING


FITTING THE BREAST SKIN FOR WEARABLE USE

Fit onto the Torso or Simulated Patient




CLEANING & STORAGE

Clean off any surface dirt with a damp,
soft cloth, using only warm water and mild detergent

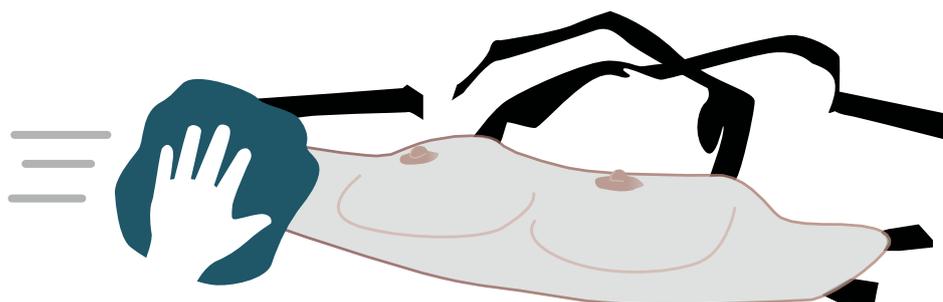

TAKE A LOOK
AT OUR **VIDEO**

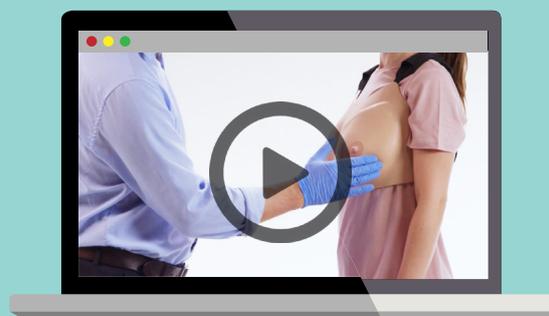

VISIT THE WEBSITE
limbsandthings.com

OR YOUTUBE CHANNEL
youtube.com/limbsandthings



Part No: 365-970

Issue 1, Sept 2021
© 2021 Limbs & Things