




USER GUIDE

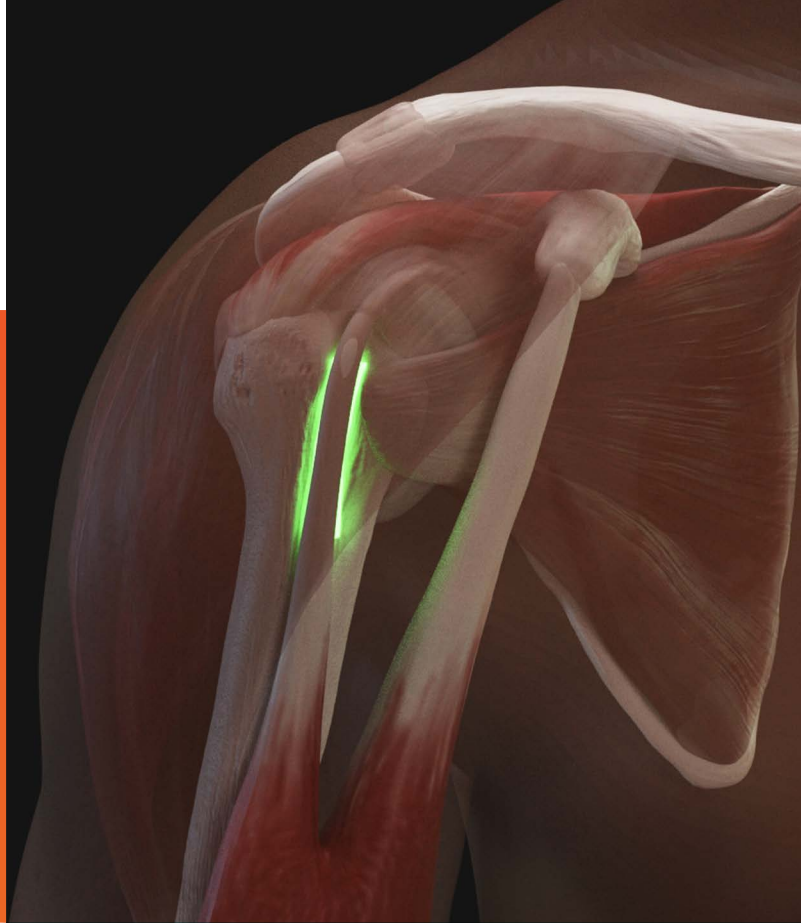

SHOULDER INJECTION TRAINER ULTRASOUND GUIDED

This product is available
in light and dark skin tones

70202

70221

FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500

INDEX

PACKAGE SUPPLIED	3
SAFETY & CLEANING INSTRUCTIONS	4
SET UP	7
• ATTACHING SHOULDER TO THE TORSO	7
• FILLING JOINTS	8
CARE & MAINTENANCE	11
• REPLACING THE SHOULDER MODULE	11

TAKE A LOOK
AT OUR **VIDEO**



VISIT THE WEBSITE
limbsandthings.com

OR YOUTUBE CHANNEL
youtube.com/limbsandthings

PACKAGE
SUPPLIED

Shoulder
Torso

70215

70226



Shoulder Module
Ultrasound Guided

70213

70224




Elbow



Needle Set

70108

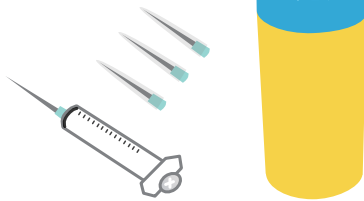

When Injecting fluid into the model please
only use the synovial fluid provided.

Filling Kit

70217




Synovial Fluid (250ml)

70022



Carry Case

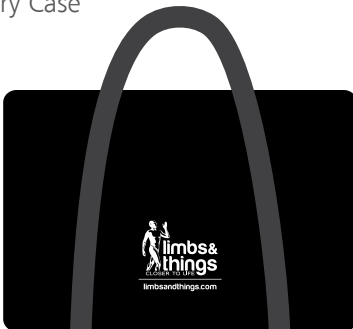

YOU WILL NEED (NOT SUPPLIED)

Linear Probe



Ultrasound Gel



SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR SHOULDER INJECTION TRAINER

DO's



Follow all instructions. Keep these instructions. Heed all warnings. Handle with care.

Use needles provided or needle of 21 gauge.



Refer all servicing to qualified service personnel. Servicing is required when the apparatus has been damaged.

Clean the product with a damp soft cloth or sponge, using only warm water with mild detergent.



DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not over rotate/move the shoulder as this can damage the product.



Do not overfill the joints. **Risk of permanent damage.**

Do not fill the model without using the filling kit (70217)




STORAGE & TRANSPORTATION



Always ensure that the products are properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.



THE ULTRASOUND SHOULDER MUST BE STORED CAREFULLY

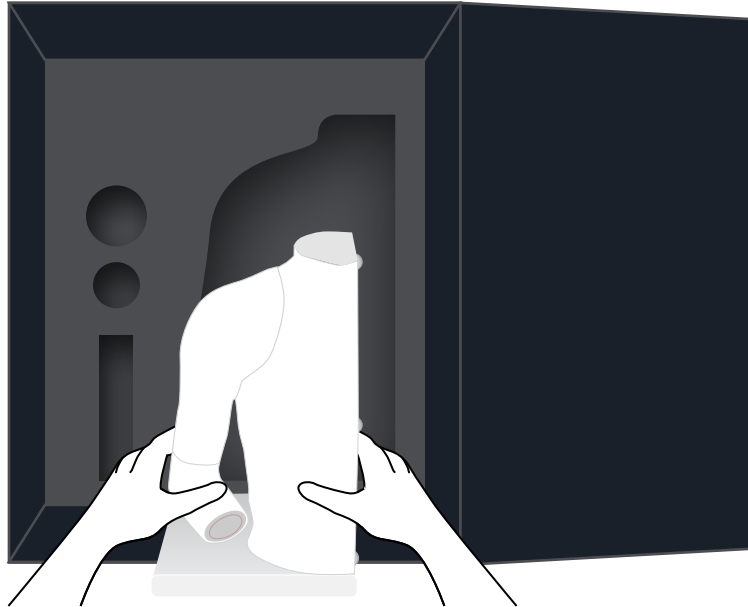

The Ultrasound Shoulder is extremely delicate, use carefully and store in its protective carry case.








THE ULTRASOUND MODULE MUST BE STORED CAREFULLY

* Applicable to 70214/70225 & 70213/70224 only


The Shoulder Ultrasound Module is extremely delicate, make sure it is stored everytime after use in its protective box.

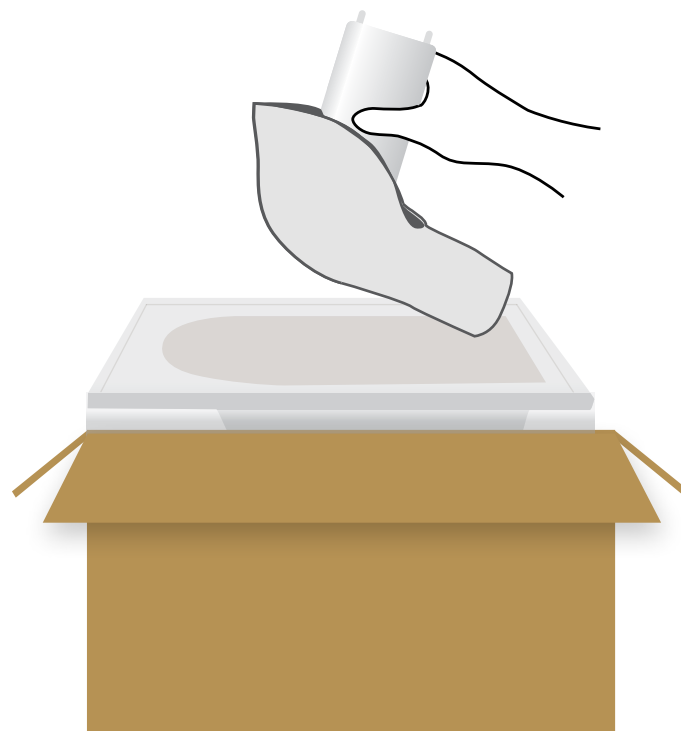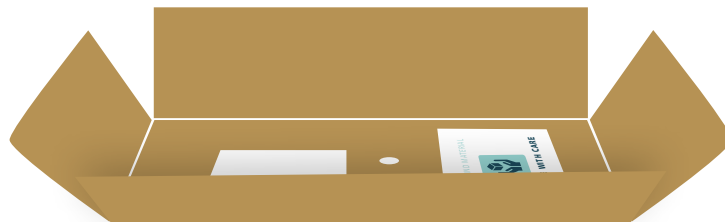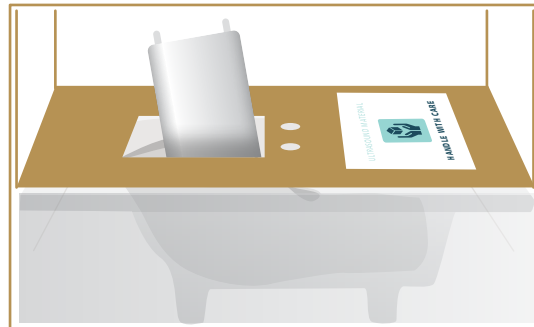


SET UP

ATTACHING SHOULDER TO THE TORSO

- 1 Attach the elbow to new module

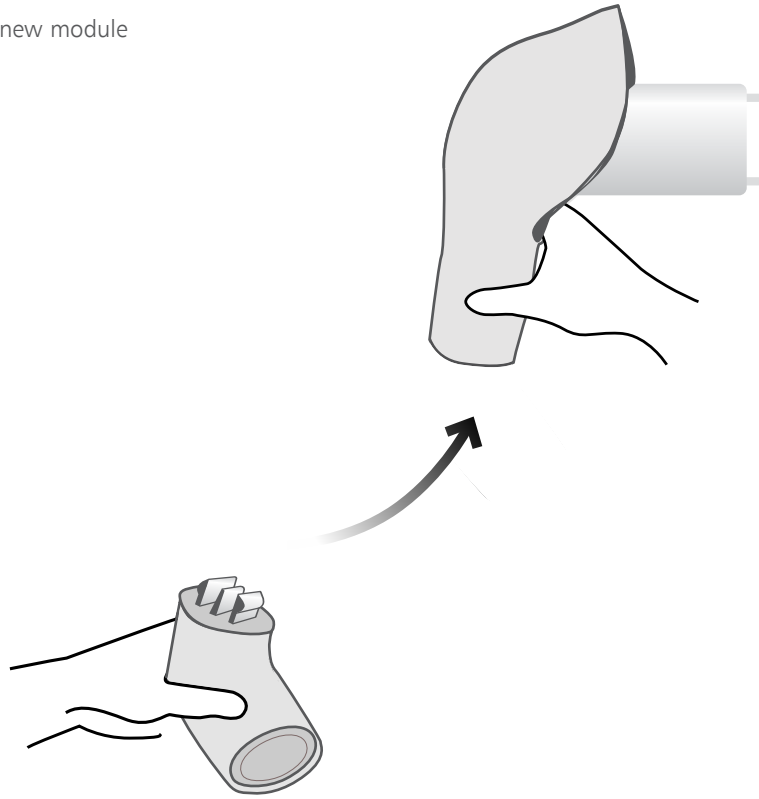

- 2 Mount shoulder into torso

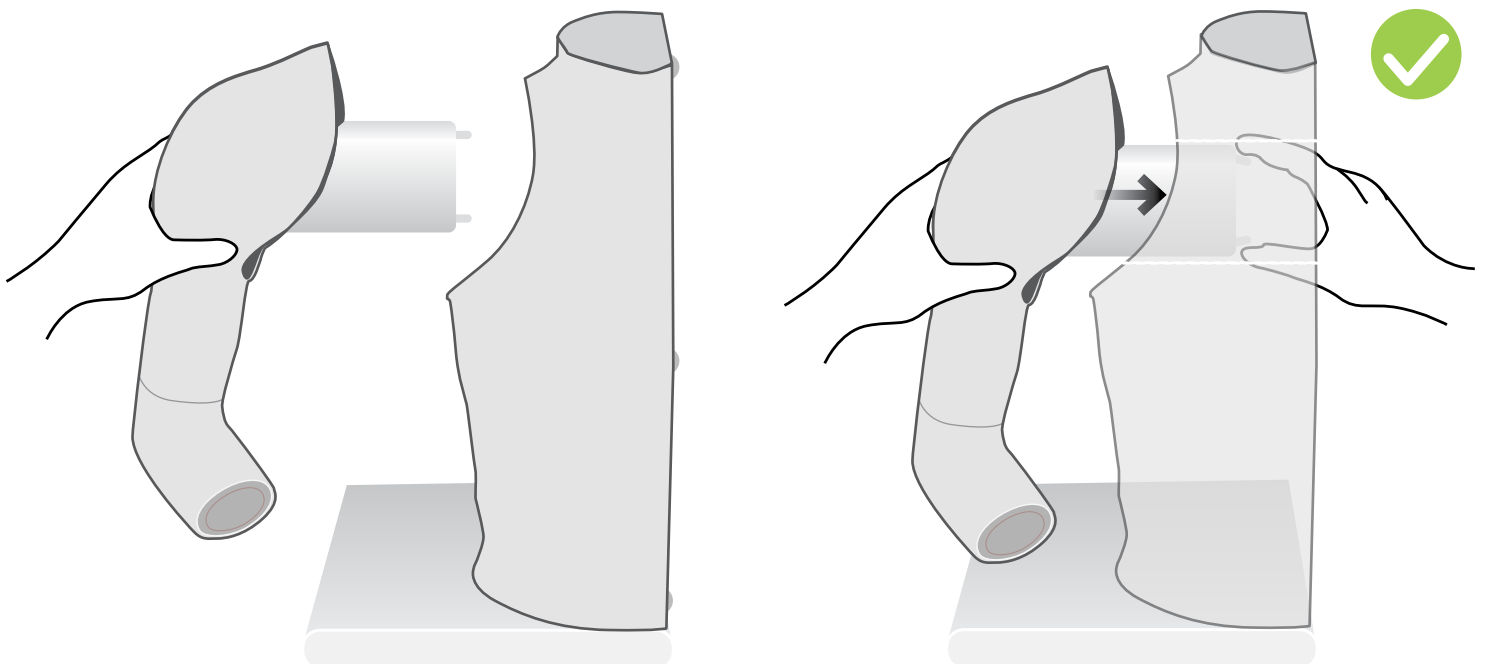

SET UP

FILLING JOINTS

- 1 Push and then screw on the tube to the "out" port



Clamp should be open



- 2 Fill 5ml syringe




ONLY USE THE SYRINGE THAT COMES IN THE FILLING SET WITH THE MODEL




SET UP


FILLING JOINTS


- 3 Push and then screw on the syringe to the "in" port

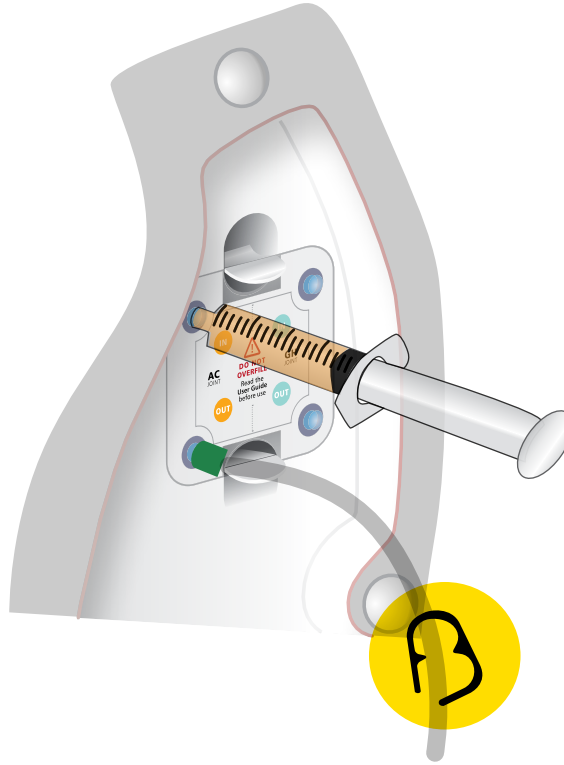

- 4 Inject fluid from syringe into joint until it starts coming out from the tube

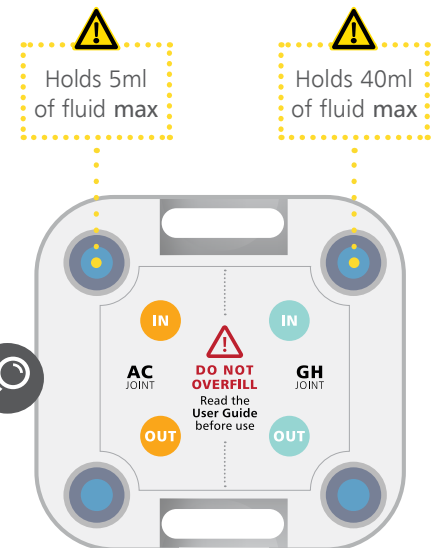


PLEASE CHECK FILLING KIT IS SET UP CORRECTLY BEFORE INTRODUCING FLUID
IF CORRECTLY SET UP THERE SHOULD BE VERY LITTLE RESISTANCE

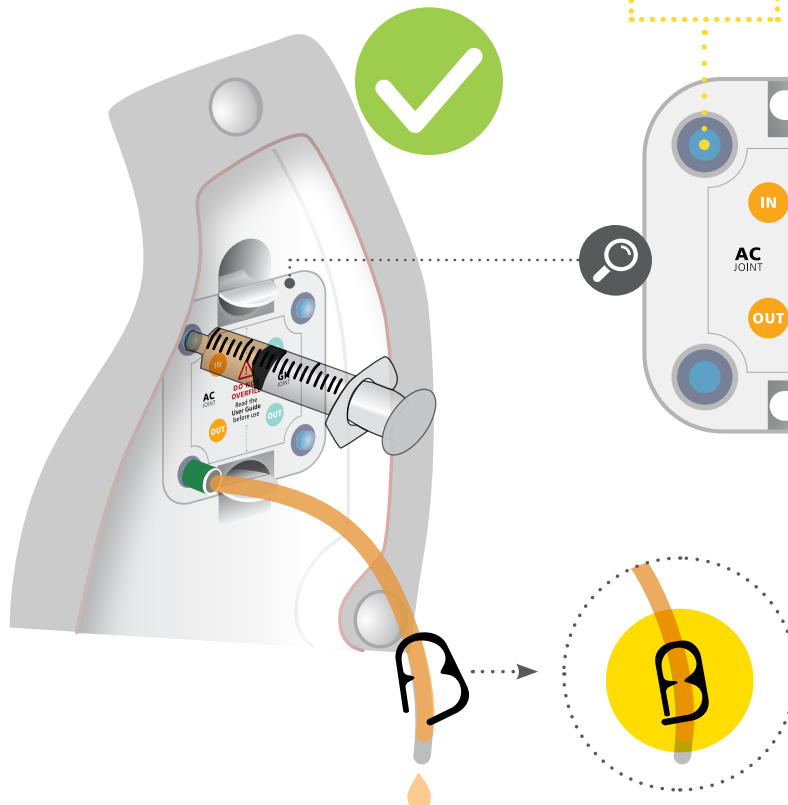





SET UP

FILLING JOINTS

ACROMIOCLAVICULAR JOINT

5ml capacity

To simulate Bursitis close the tube and add another 5ml of fluid

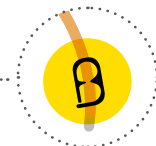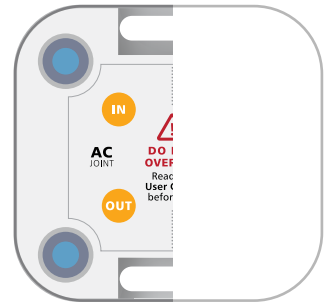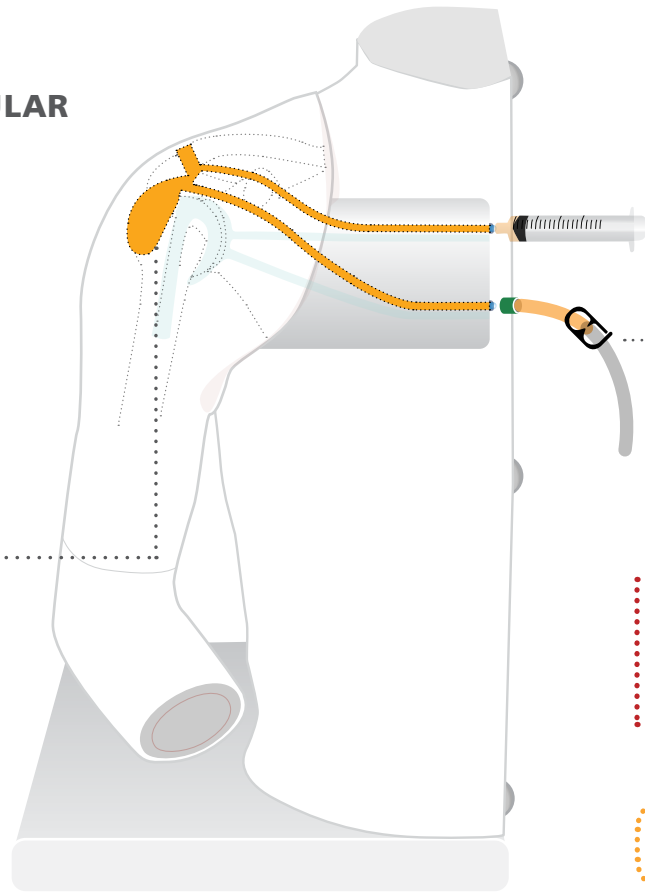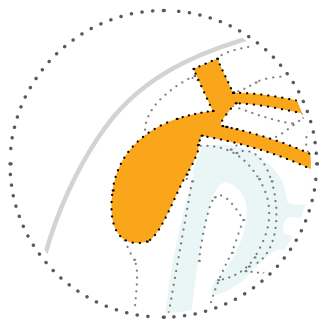


DO NOT EXCEED 5ml
WITH CLAMP CLOSED
Risk of permanent damage



Release clamp after use


GLENOHUMERAL JOINT

40ml capacity.

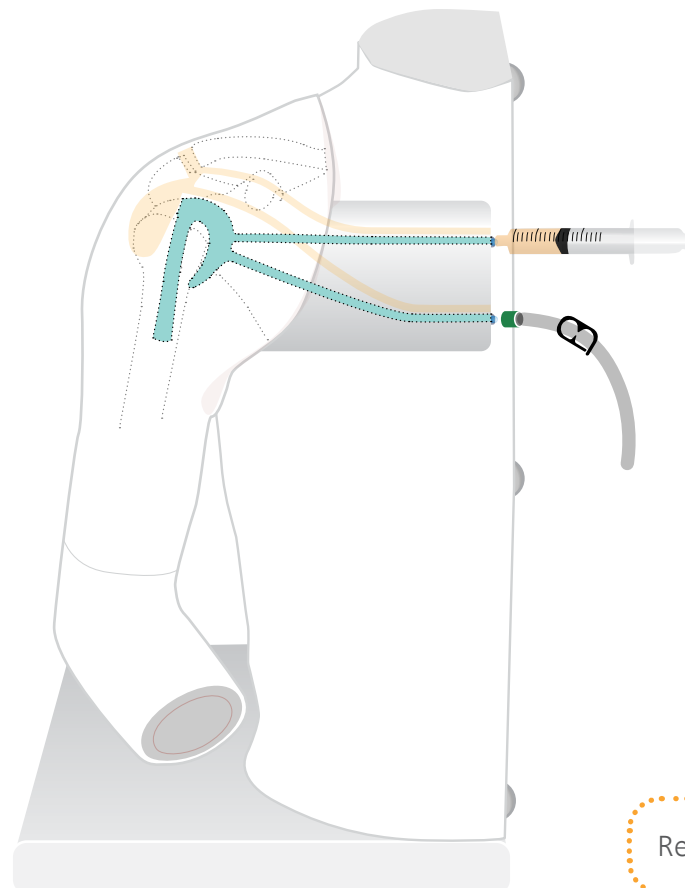


Release clamp after use


1 Remove the filling tubes

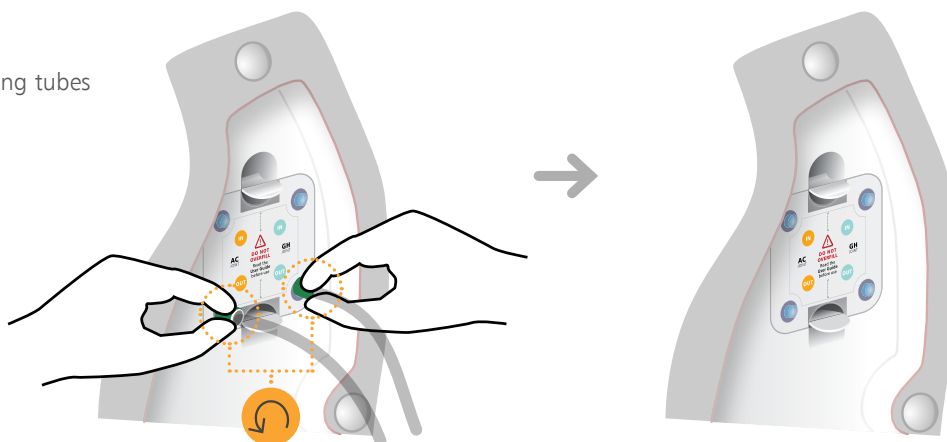

2 Remove shoulder from torso

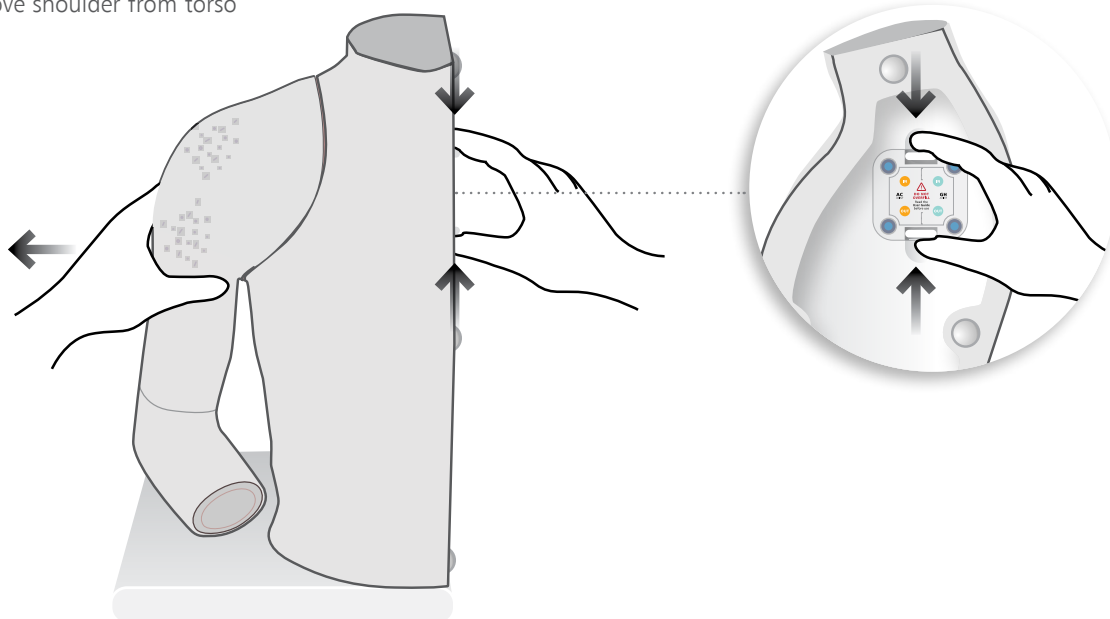

3 Remove the elbow

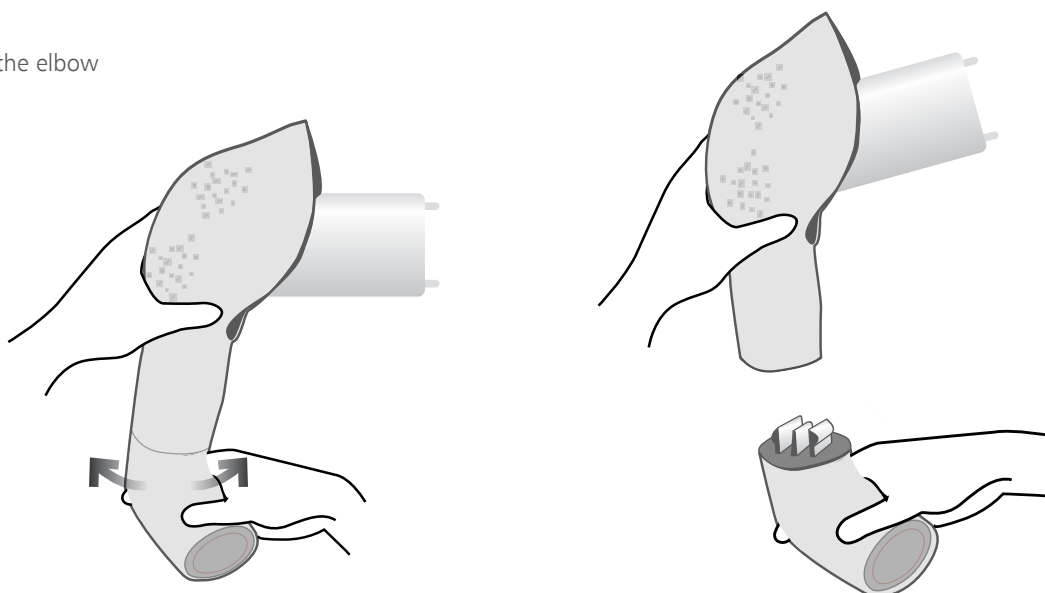

YOU MAY
ALSO BE
INTERESTED IN...

HAND & WRIST INJECTION TRAINER

70230



KNEE ASPIRATION & INJECTION TRAINER WITH ULTRASOUND CAPABILITY

70103 70114

