



## USER GUIDE



# MEDIOLATERAL PERINEAL REPAIR TRAINER

This product is available in light and dark skin tones

61141

61151

FOR MORE SKILLS TRAINING PRODUCTS VISIT

[limbsandthings.com](https://limbsandthings.com)

Sussex Street, St Philips, Bristol, BS2 0RA, UK | [sales@limbsandthings.com](mailto:sales@limbsandthings.com) | +44 (0) 117 311 0500


## INDEX

PACKAGE SUPPLIED	3
SAFETY & CLEANING INSTRUCTIONS	4
SET UP	5
• SECURING THE STAND	5
• FITTING PERINEAL REPAIR PAD ON TO THE STAND	6
CARE & MAINTENANCE	8

## TAKE A LOOK AT OUR **VIDEO**

VISIT THE WEBSITE  
[limbsandthings.com](https://limbsandthings.com)

OR YOUTUBE CHANNEL  
[youtube.com/limbsandthings](https://youtube.com/limbsandthings)

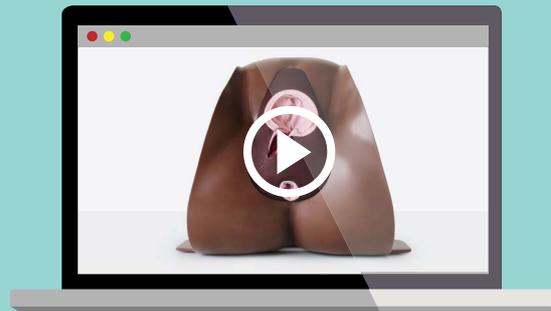

SCAN ME

## PACKAGE SUPPLIED

Mediolateral Perineal Repair Pad (x2)

61146

61156



Perineal Repair Stand with Clamps

61109

61129



Perineal Repair Illustration Pack

61112

61132



Perineal Repair Block Loading Handle

61110




## YOU'LL NEED (NOT SUPPLIED)

Sutures



Forceps & Relevant Instruments

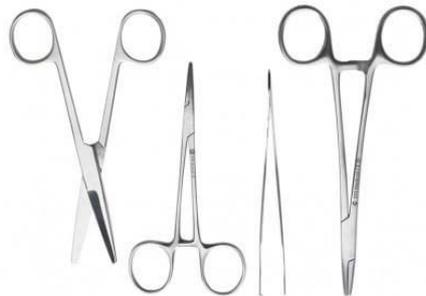

*(We recommend using non-tooth forceps only to minimise damage to the repair pad)*

# SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR PERINEAL REPAIR TRAINER

## DO's



Follow all instructions. Keep these instructions. Heed all warnings.

Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.



Handle with care. The silicone is soft and can mark easily.

Remove sutures upon completing a repair. Great care must be taken to ensure that the pad itself isn't damaged. We recommend using small sharp scissors to remove sutures.



## DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not use any attachments other than those supplied by the manufacturer.



Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.

Do not allow young children to use this product.



## TRANSPORTATION




Always ensure that the product is properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.

1



Do not over tighten


2

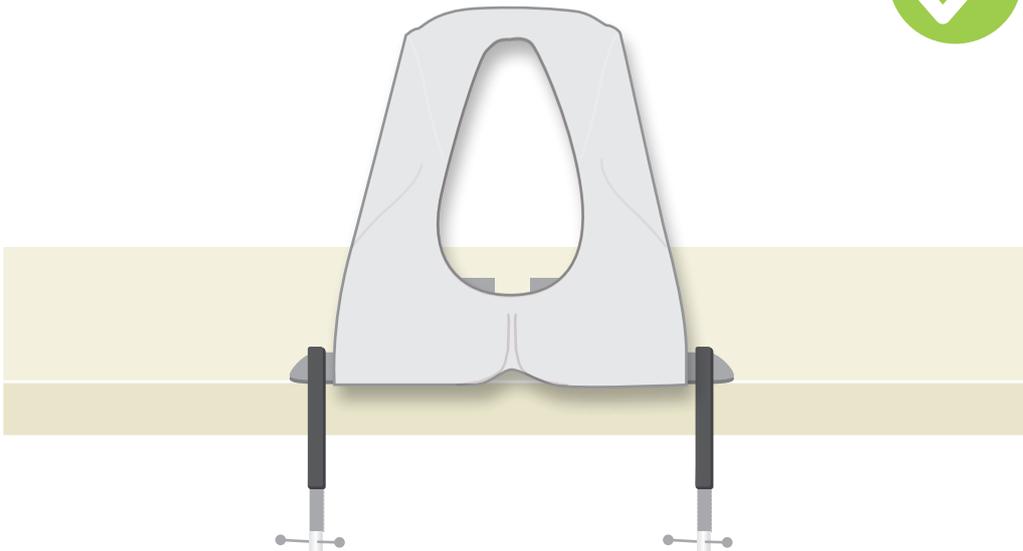

SET UP

FITTING PERINEAL REPAIR BLOCK ON TO THE STAND


1

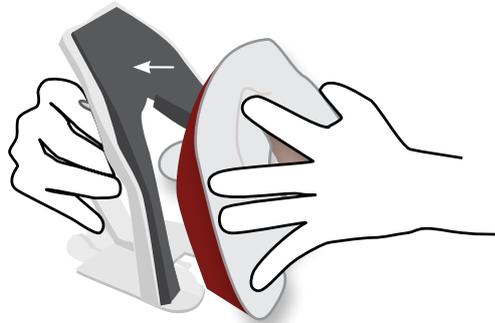

2



3

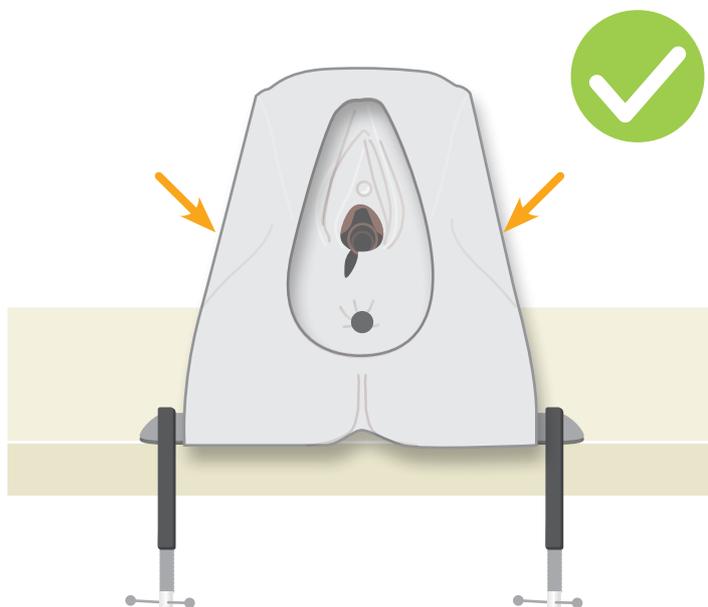



YOU MAY  
ALSO BE  
INTERESTED IN...

MIDLINE PERINEAL REPAIR  
PAD (x2)

61145

61155



ANAL SPHINCTER  
REPAIR BLOCK (x2)

61107

61127



EPISIOTOMY TRAINER MODULE

61104

61124



STAND NOT INCLUDED