




USER GUIDE

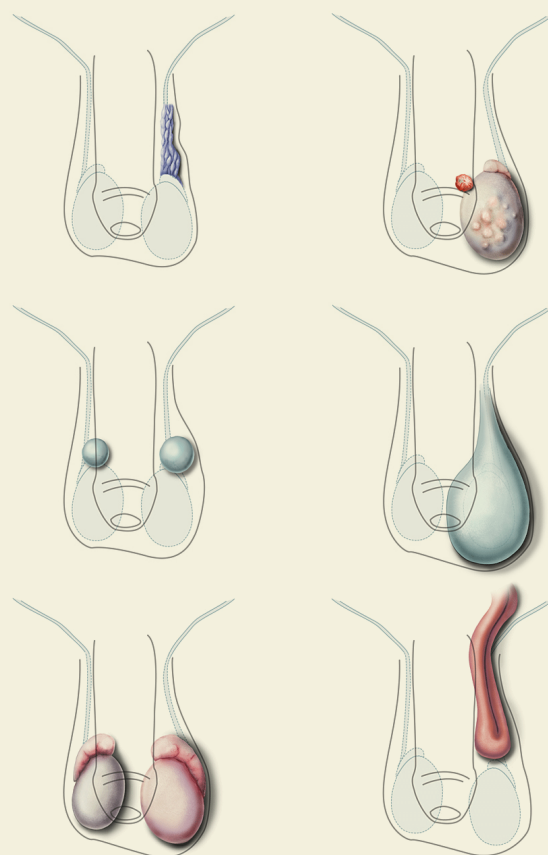

CLINICAL MALE PELVIC TRAINER (CMPT)

ADVANCED



This product is available
in light and dark skin tones

60951

60965

FOR MORE SKILLS TRAINING PRODUCTS VISIT


limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500

INDEX

PACKAGE SUPPLIED	3
SAFETY & CLEANING INSTRUCTIONS	4
SET UP	5
• SETTING UP THE CMPT	5
• ATTACHING THE FORESKIN	6
TRAINING	7
• STANDING & SUPINE	7
CLEANING & STORAGE	7
• CLEANING & STORAGE AFTER USE	7
AUGMENTED REALITY TRAINING (ART)	8
• SAFETY & CLEANING INSTRUCTIONS	8
• ART SET UP	9

TAKE A LOOK
AT OUR **VIDEO**

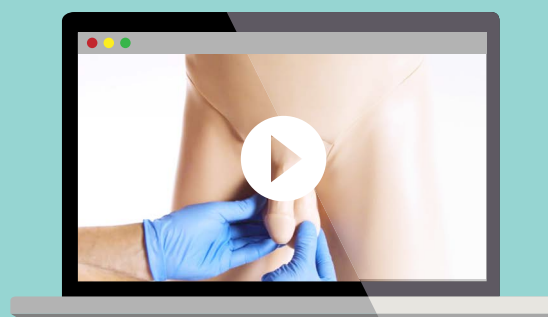

VISIT THE WEBSITE
limbsandthings.com

OR YOUTUBE CHANNEL
youtube.com/limbsandthings

PACKAGE SUPPLIED

CMPT Abdominal Insert

60959

60973



Lower Torso



Base



Carry Case



LED Pen Torch



ART Mat


60135



CMPT Foreskins (x4)


60961

60974

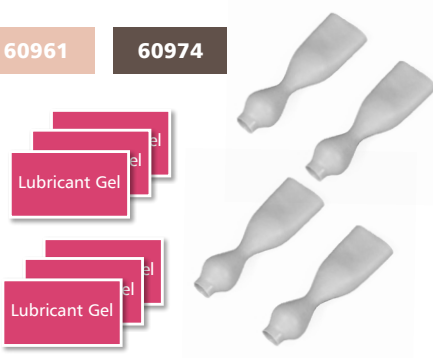

CMPT Pubic Bone (x2)


60960

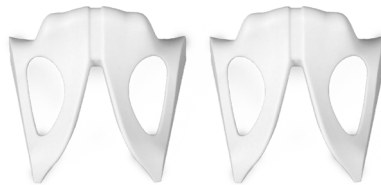

CMPT Module 1: Normal

60952


60966

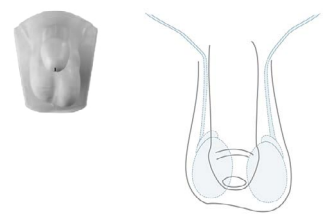

CMPT Module 2: Varicocele

60953


60967

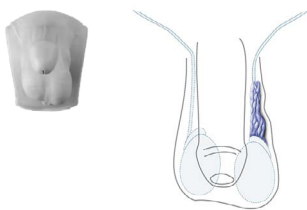

CMPT Module 3: Tumours

60954


60968

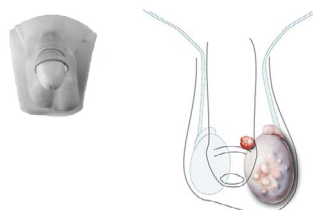

CMPT Module 4: Epididymal Cysts

60955


60969

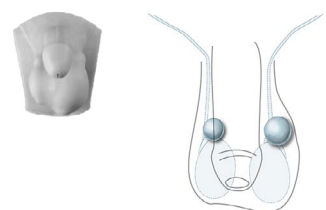

CMPT Module 5: Hydrocele

60956


60970

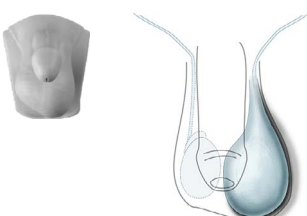

CMPT Module 6: Epididymo-orchitis

60957


60971

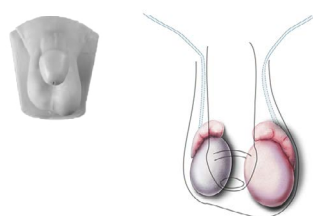

CMPT Module 7: Indirect Inguinal Hernia

60958

60972

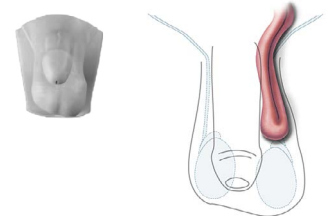

SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR CLINICAL MALE PELVIC TRAINER

DO's



Follow all instructions.
Keep these instructions.
Heed all warnings.

Only use water-based lubricant sachets, such as the ones supplied with this model.



Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.

Clean the product with a damp soft cloth or sponge, using only warm water with mild detergent, after every training session.



DON'Ts




Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not use any attachments other than those supplied by the manufacturer.



Do not wear any jewellery items.

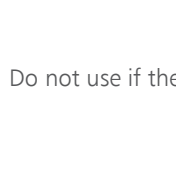

Do not use if the skin is torn.



Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.

STORAGE & TRANSPORTATION




Always ensure that the products are properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.

SET UP


SETTING UP THE CMPT

1 Select CMPT Module

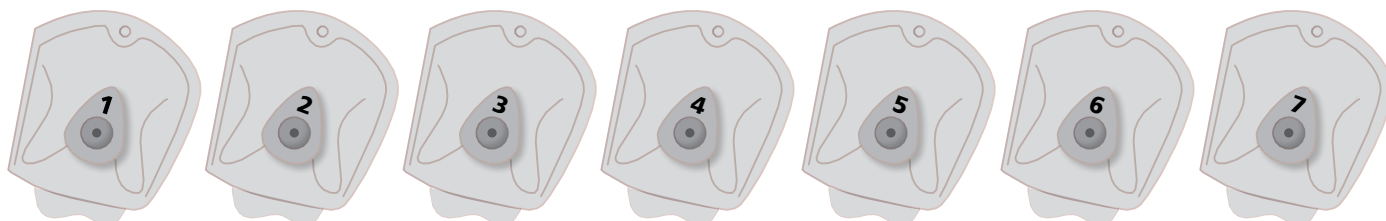


CMPT Modules - Rear view

2 Removing/Attaching CMPT Module to Pubic Bone

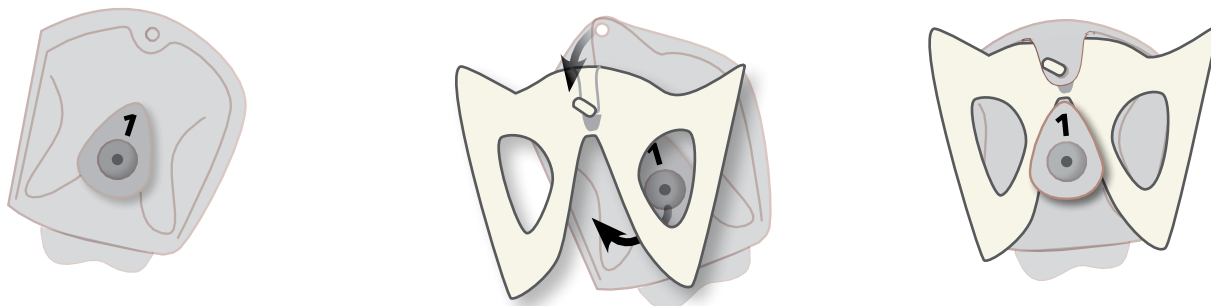

3 Fitting CMPT Module and Pubic Bone assembly in the Lower Torso

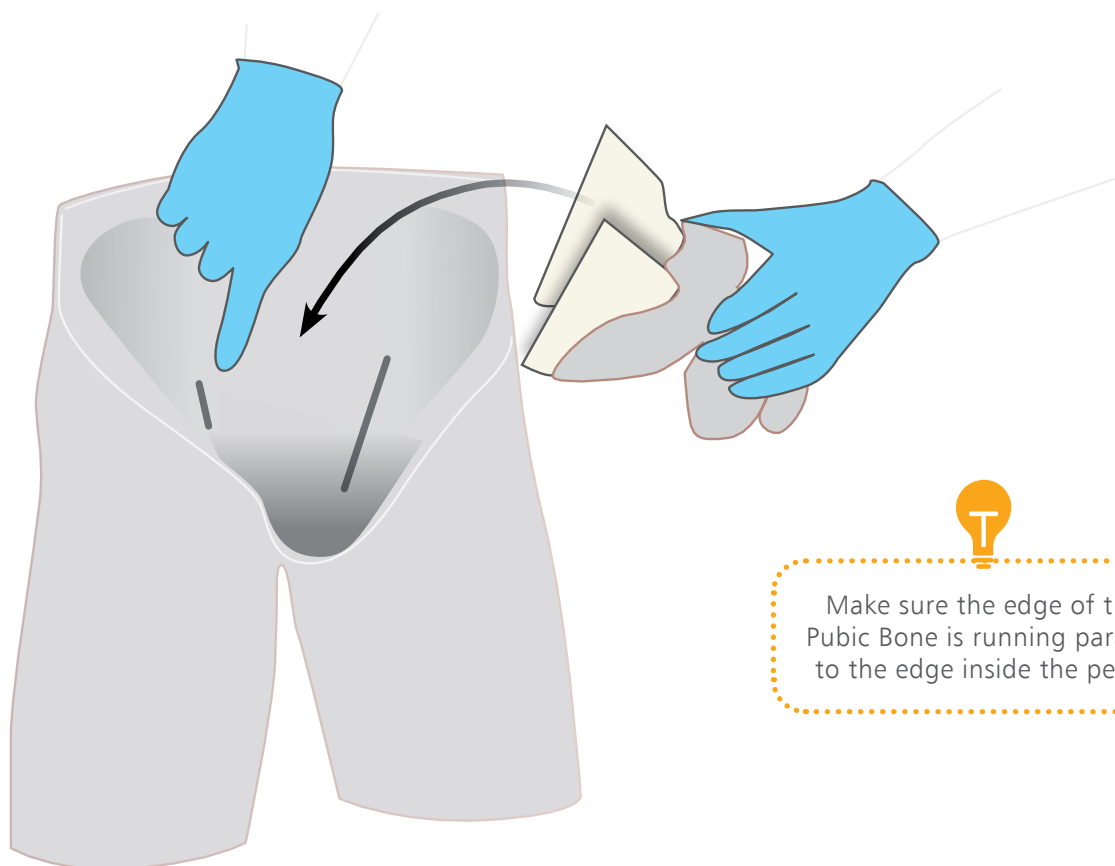



Make sure the edge of the Pubic Bone is running parallel to the edge inside the pelvis

SET UP




SETTING UP THE CMPT

3

Removing/Attaching Abdominal Insert

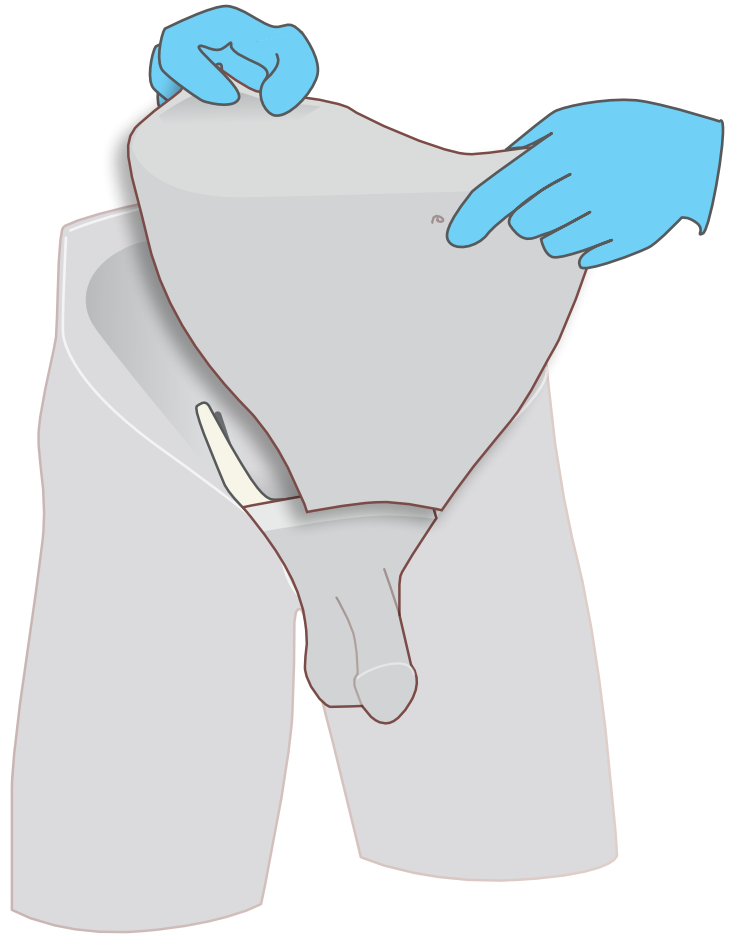
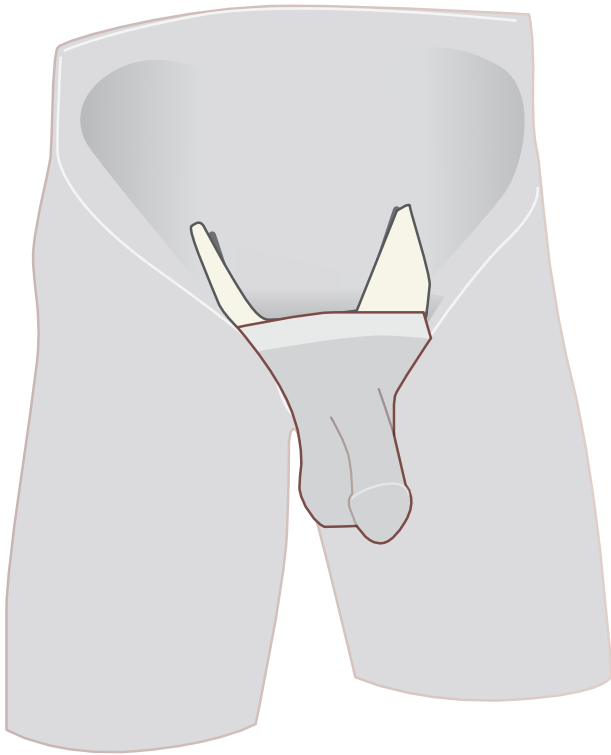


ATTACHING THE FORESKIN

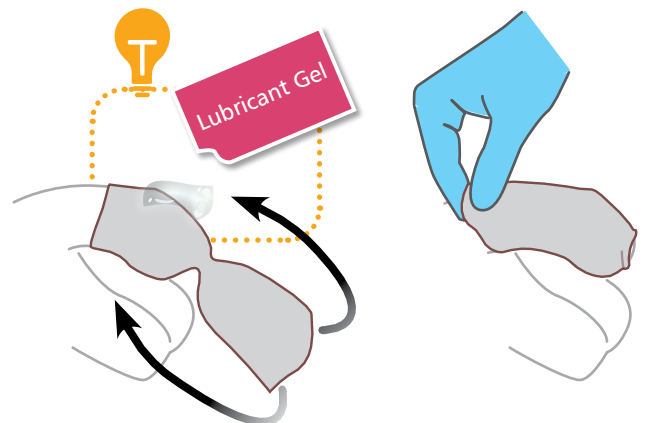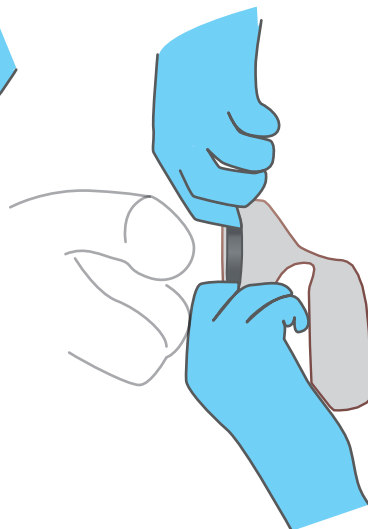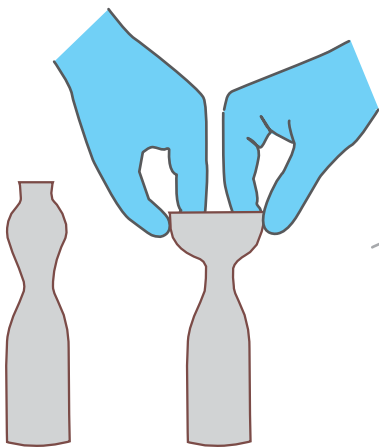


TRAINING

Can be used in 2 positions: standing & supine

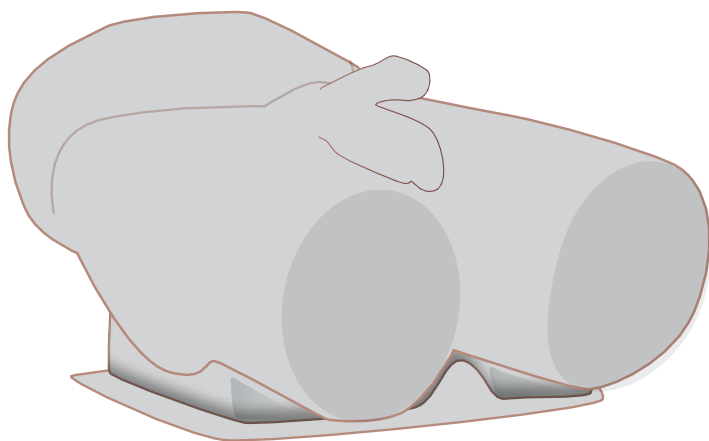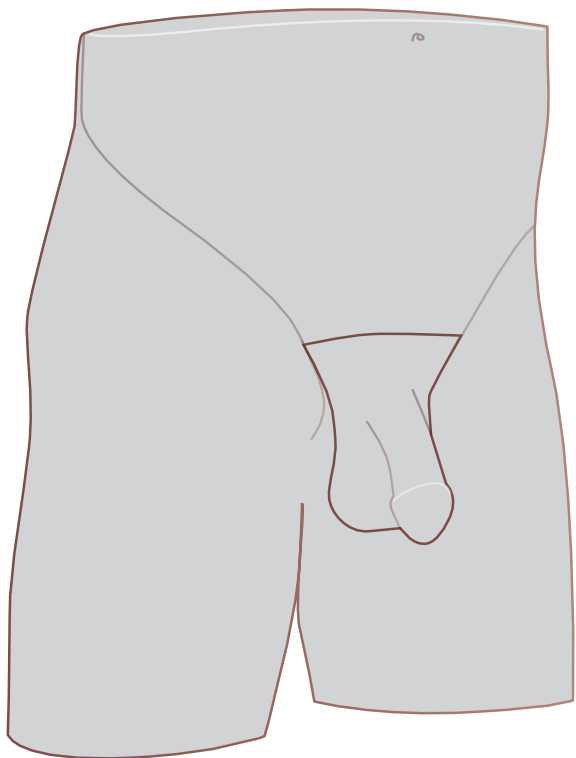

CLEANING & STORAGE

Clean off any surface dirt with a damp, soft cloth,
using only warm water and mild detergent

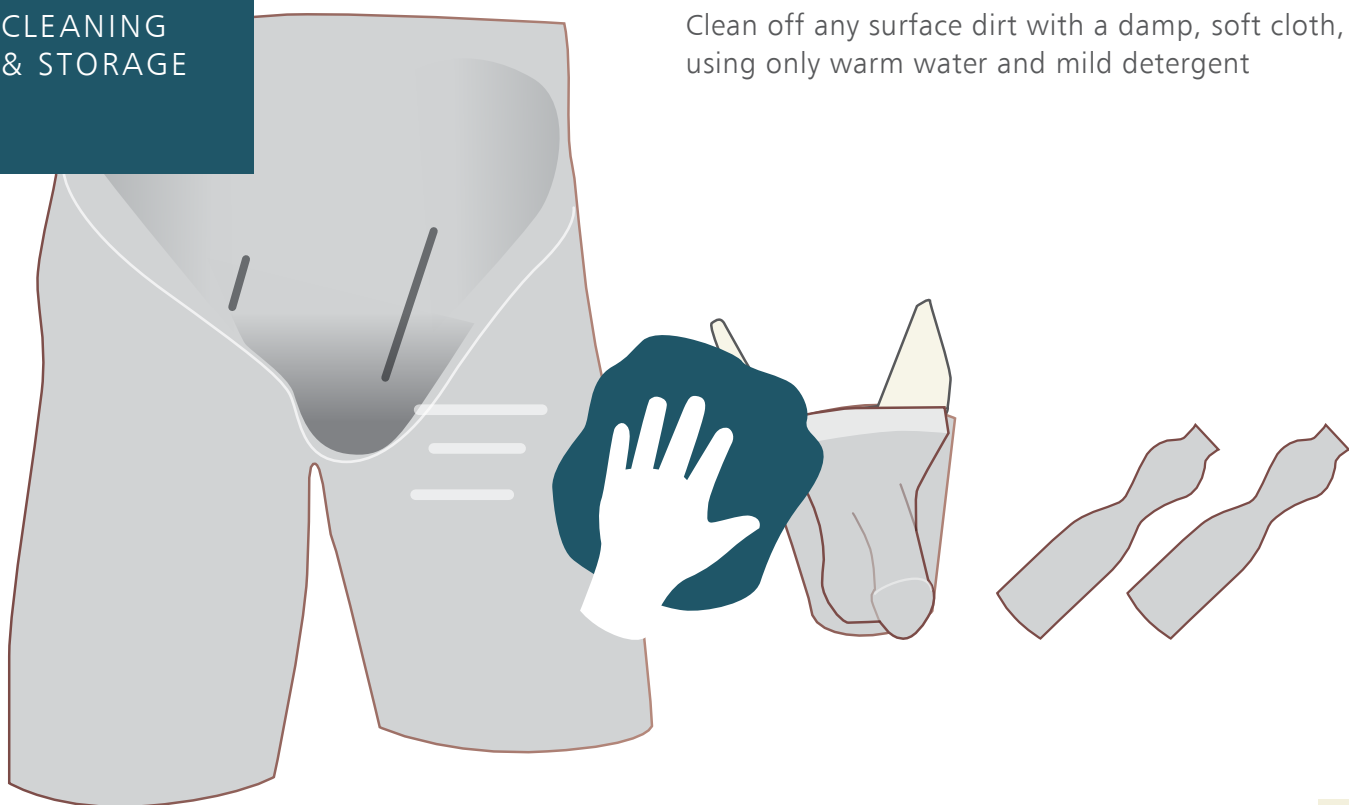



AUGMENTED REALITY TRAINING (ART)

for your CMPT Simulator

SAFETY & CLEANING INSTRUCTIONS

Read the safety instructions
carefully before using your
ART Mat

DO's



Follow all instructions.
Keep these instructions.
Heed all warnings

Use mat on flat surface



Only use attachments/accessories
specified by Limbs & Things

Refer all servicing to qualified service
personnel. Servicing is required when
the apparatus has been damaged



DON'Ts



Do not install near any heat sources,
do not expose the product to flames
or use near naked flame sources.
Do not use flammable solvents near
or on this product

Do not set up the product near
electrical appliances or items
damaged by water



Do not cover the graphic of the mat
with other items

Do not move the mat with a
model on top of it



STORAGE



Store rolled up with print on the
outside



Don't fold



Don't place heavy objects on top

CLEANING



Can be cleaned with water, soap and
non-alcoholic cleaning agents



Avoid abrasive and
alcoholic cleaning agents




Make sure mat is dry before storing

ART SET UP


1

Take the mat out of tube

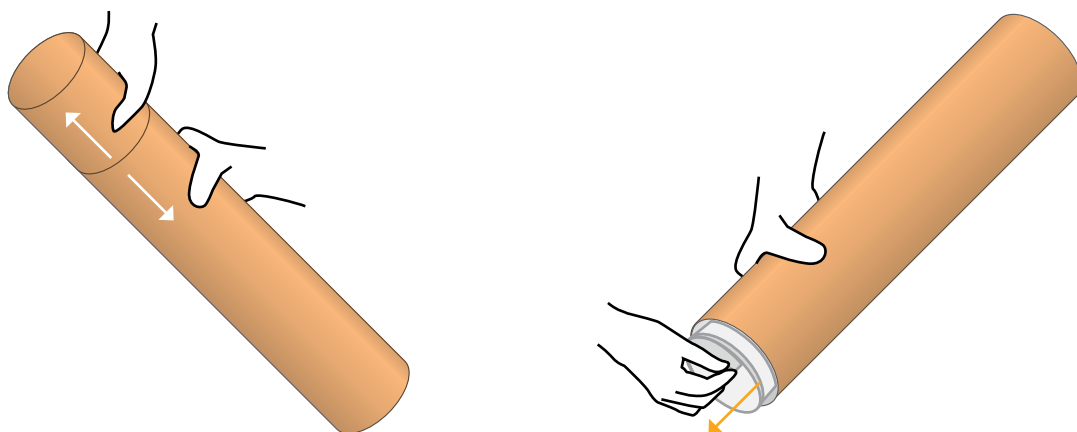

2



Place the mat on a table

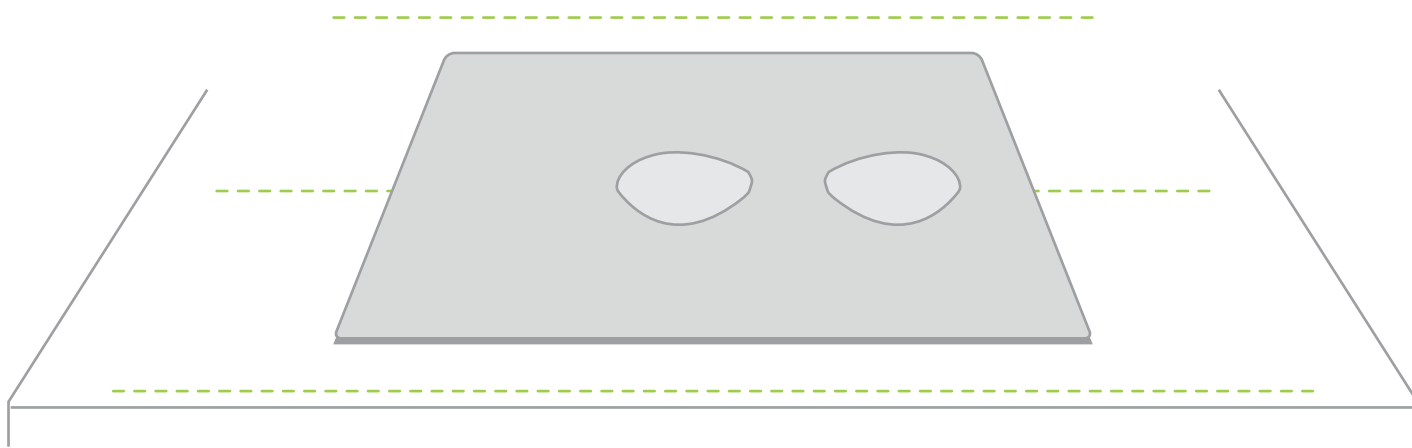

Note: Diagrams are for illustrative purposes, your product may differ from the example shown



3

Place the model on the designated area of the mat

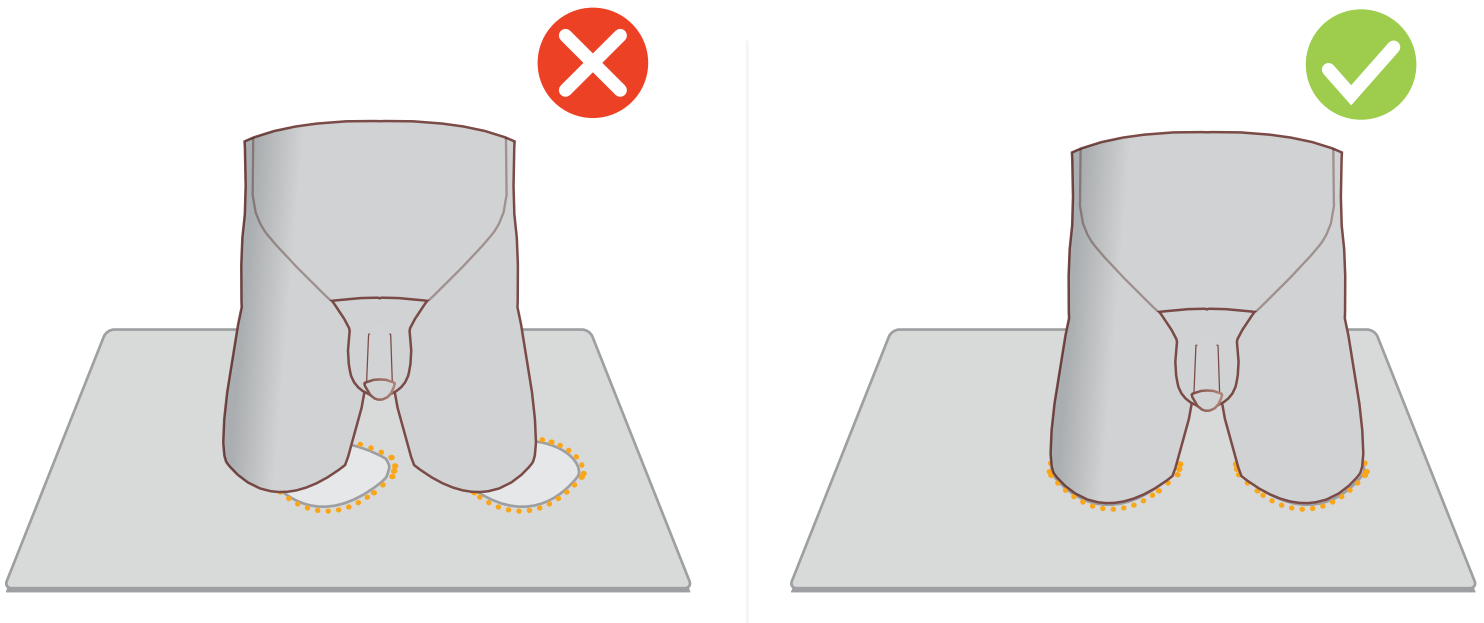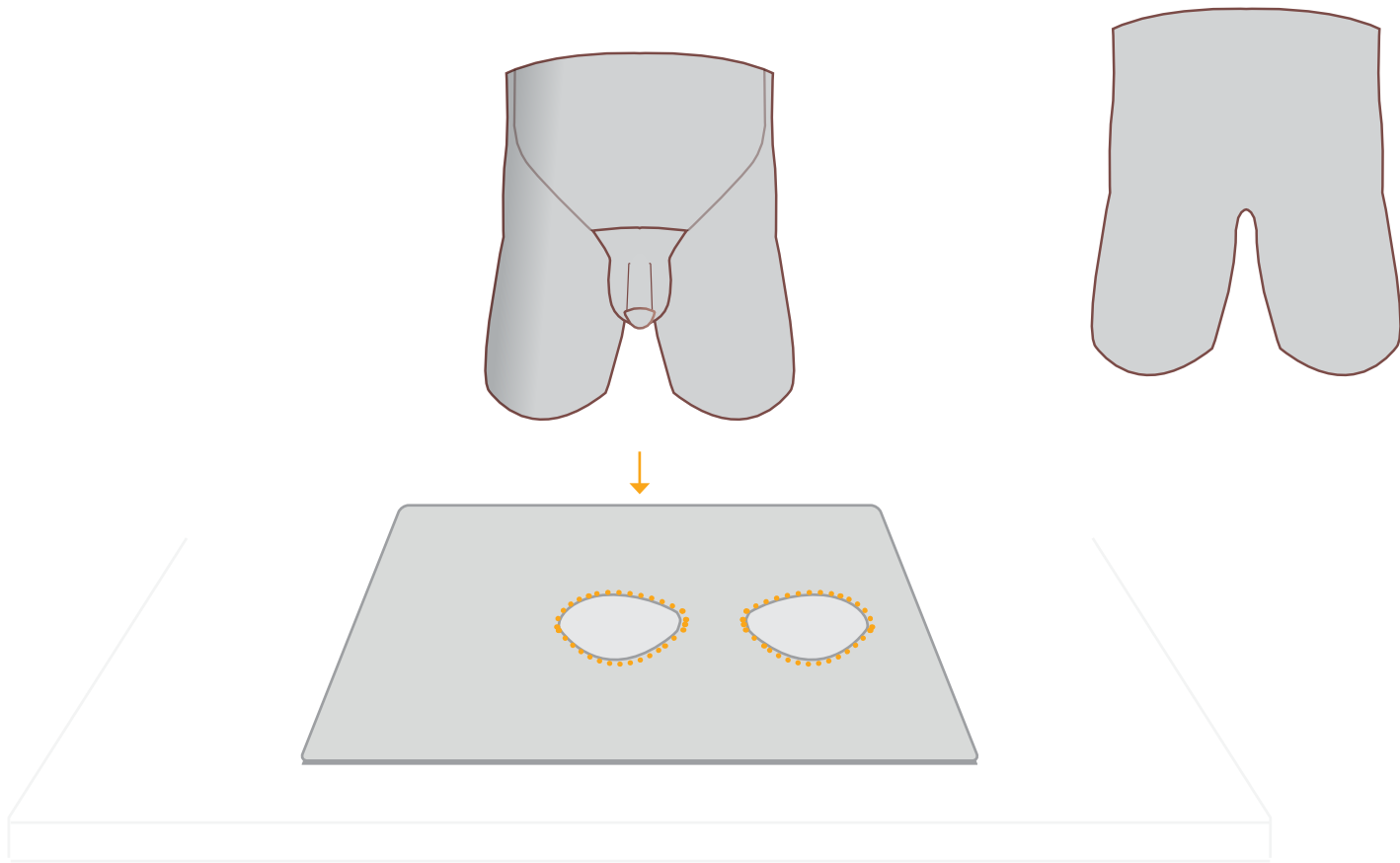

Note: Diagrams are for illustrative purposes, your product may differ from the example shown

Use the A.R.T. app to scan the mat

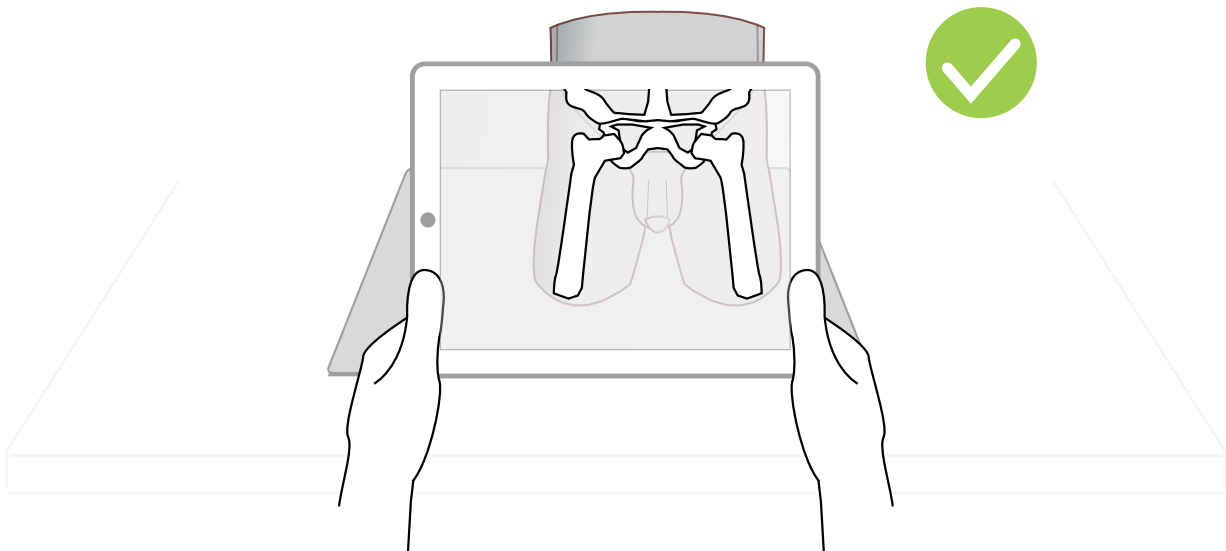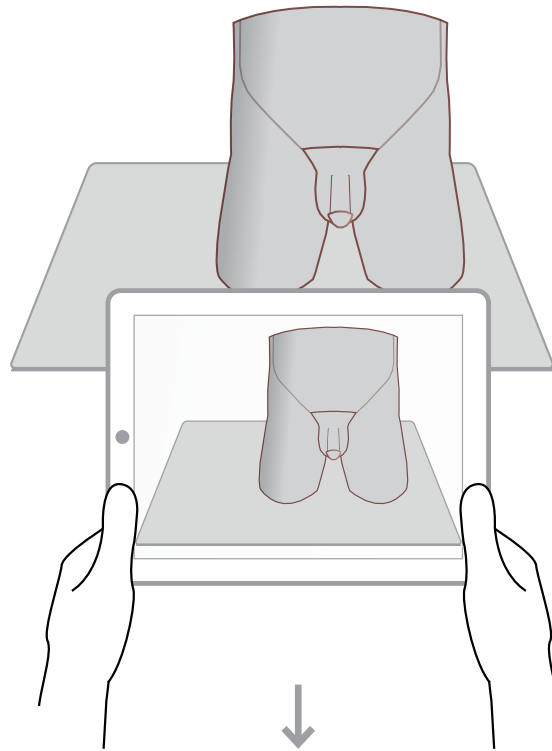

For app troubleshooting please visit the website.

Note: Diagrams are for illustrative purposes, your product may differ from the example shown

YOU MAY
ALSO BE
INTERESTED IN...

MALE RECTAL
EXAMINATION TRAINER
ADVANCED

60171

60187



limbs&
things

Part No: 365-001

Issue 4, May 2025

© 2025 Limbs & Things



FM 818255