



USER GUIDE



ASSISTED VAGINAL BIRTH (AVB) MODULE - PROMPT FLEX

This product is available
in light and dark skin tones

80310

80410

FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500

INDEX

PACKAGE SUPPLIED	3
SAFETY & CLEANING INSTRUCTIONS	4
SET UP	5
• GETTING THE PROMPT FLEX READY	5
- Removing the ABDOMINAL SKIN	5
- Removing PELVIC RING CLAMP, CERVIX & PERINEUM	6
• SETTING THE MODULE UP FOR FORCEPS	7
• SETTING THE MODULE UP FOR VENTOUSE	10
CLEANING & STORAGE	14

TAKE A LOOK AT OUR
PROMPT FLEX VIDEO



VISIT THE WEBSITE
limbsandthings.com

OR YOUTUBE CHANNEL
[youtube.com/limbsandthings](https://www.youtube.com/limbsandthings)

PACKAGE SUPPLIED

Perineum for AVB

80312

80412



Foam uterus support for ECSM

80175



Uterus for AVB

80311

80411



Baby mask for AVB

80313

80413



Foam Inserts for AVB

80314



YOU'LL NEED (NOT SUPPLIED)

Birthing Simulator - PROMPT Flex

80100

80200

80106

80206



SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR ASSISTED VAGINAL BIRTH MODULE - PROMPT FLEX

DO's



Follow all instructions. Keep these instructions. Heed all warnings.

Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.



Use L&T branded lubricant with this product. Use of non-recommended fluid products may weaken or damage the simulator.

Always lubricate the uterus, birth canal and baby before each use.



Clean the product with a soft damp cloth or sponge, using only warm water with mild detergent, after every training session.

DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near a naked flame source. Do not use flammable solvents near or on this product.

Do not use any attachments other than those supplied by the manufacturer.



Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.

Do not allow young children to use this product.



STORAGE & TRANSPORTATION






Always ensure the product is carefully packed and secured during transportation and storage in order to prevent personal injury or damage to the product.

SET UP

GETTING THE PROMPT FLEX READY





Removing the ABDOMINAL SKIN

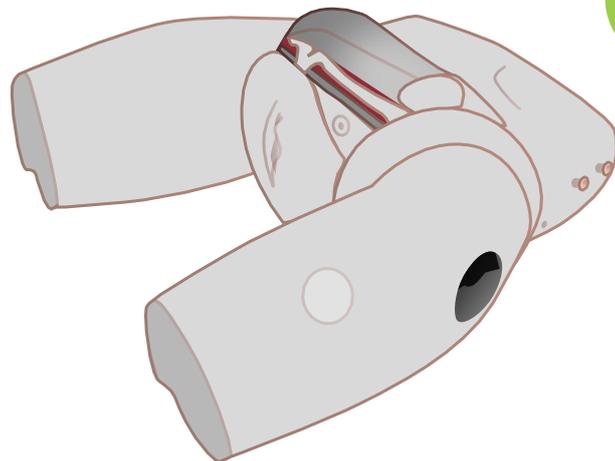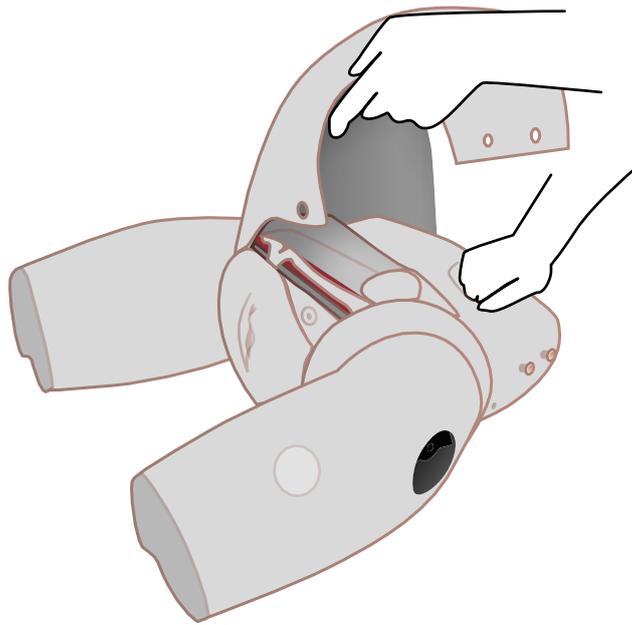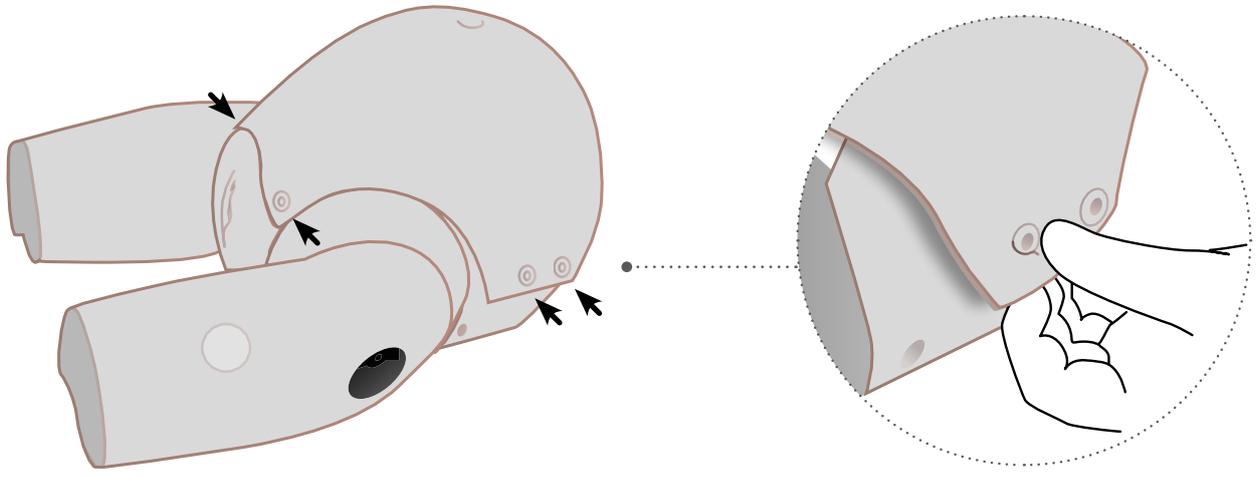

SET UP




GETTING THE PROMPT FLEX READY

Removing PELVIC RING CLAMP, CERVIX & PERINEUM

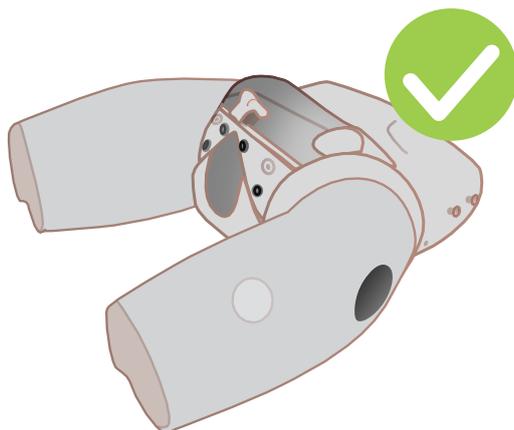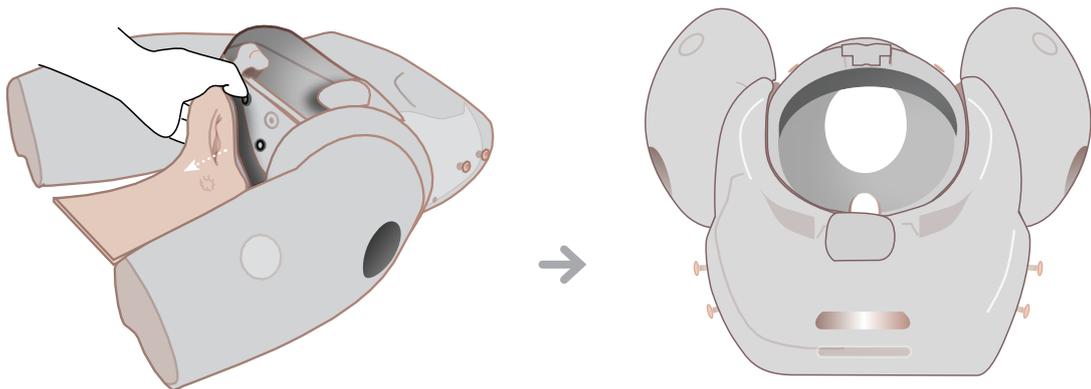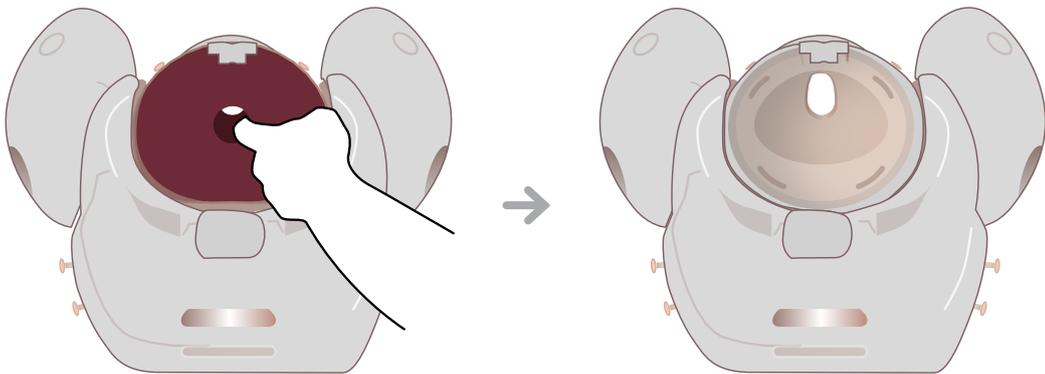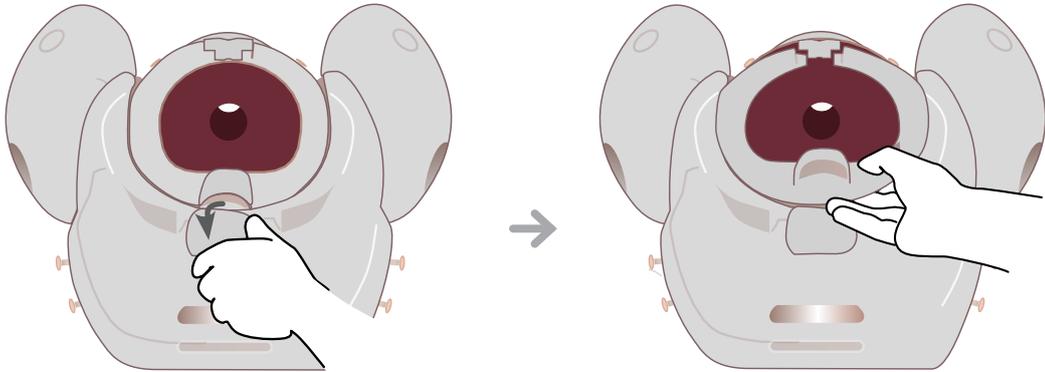






SET UP

SETTING THE MODULE UP FOR FORCEPS

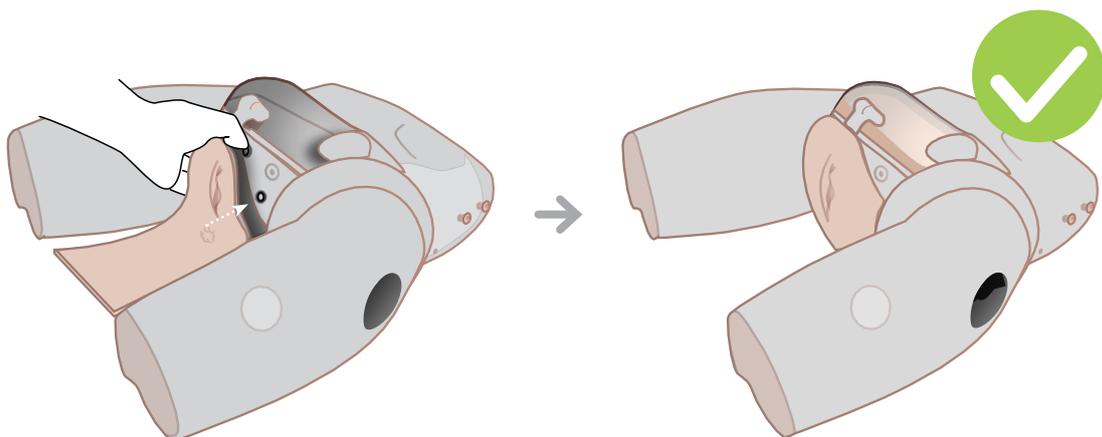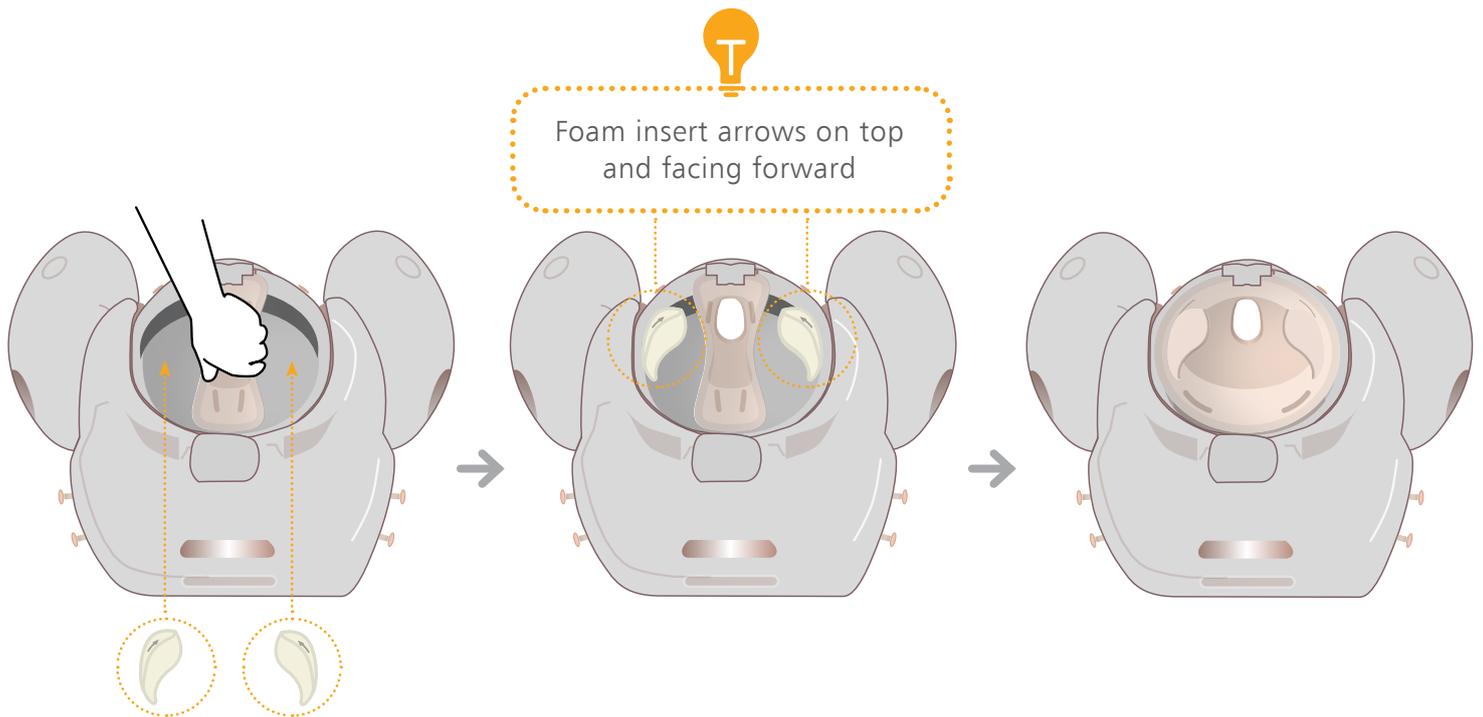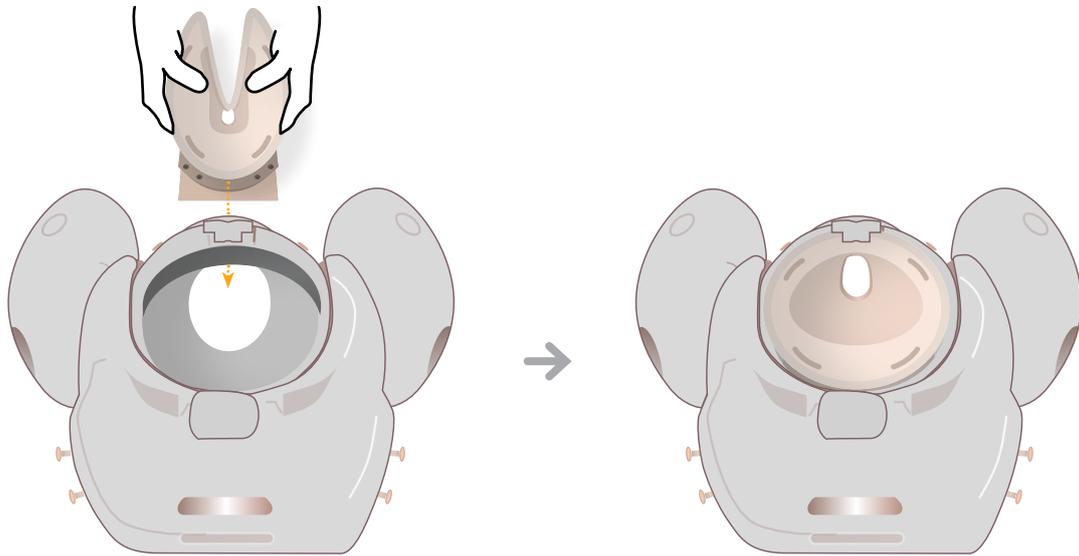




SET UP

SETTING THE MODULE UP FOR FORCEPS

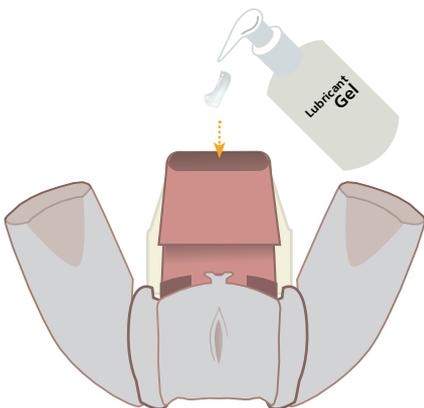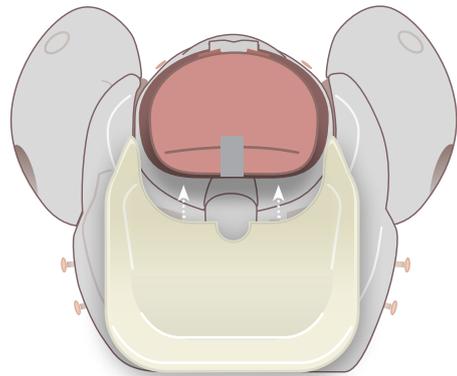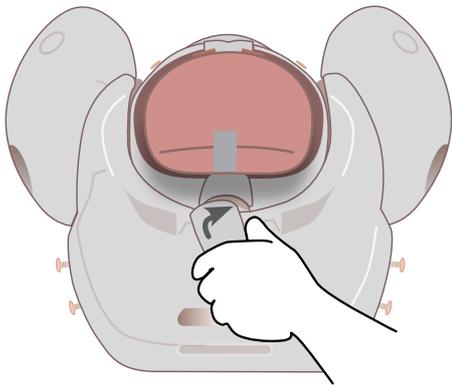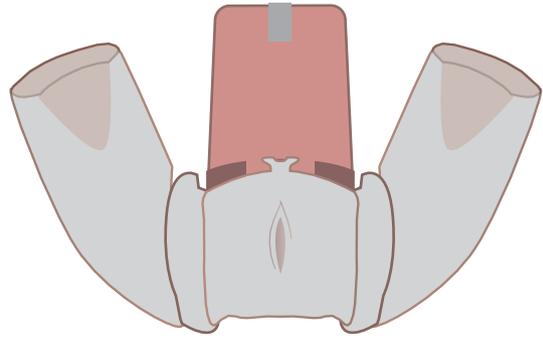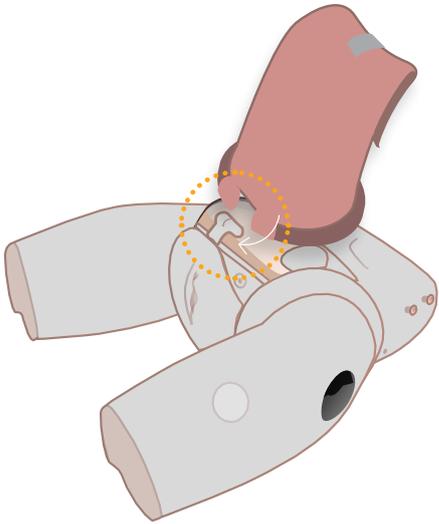



SET UP

SETTING THE MODULE UP FOR FORCEPS

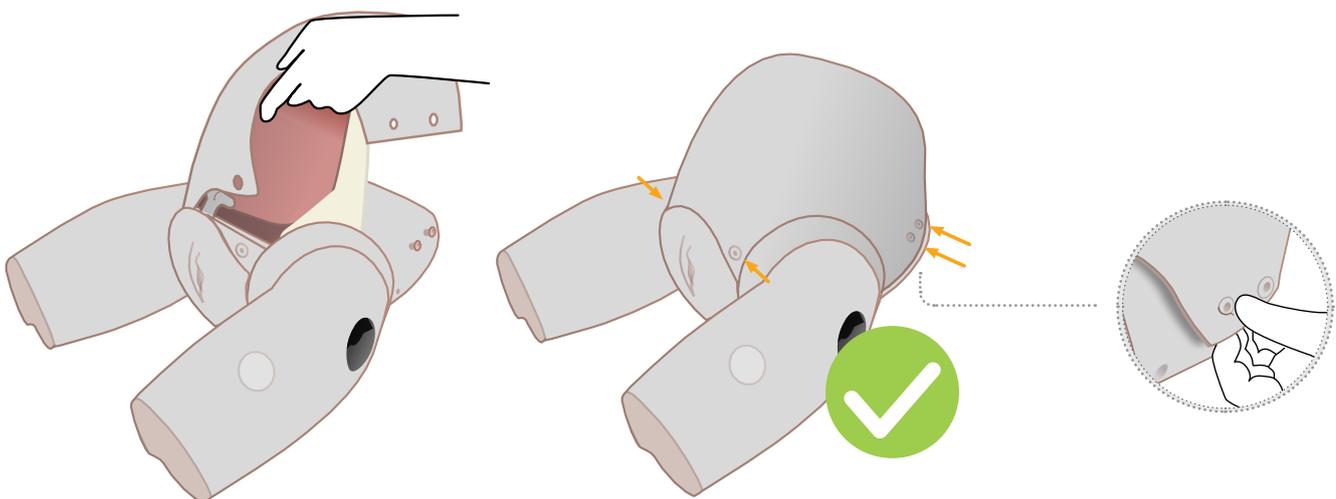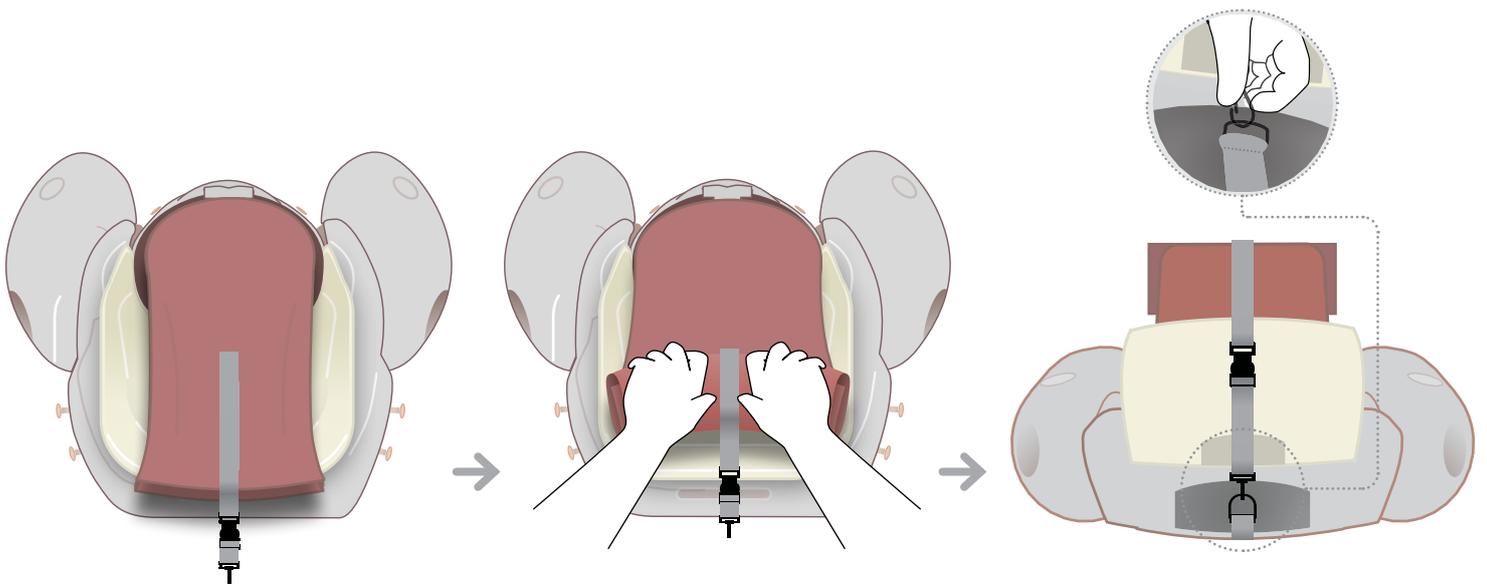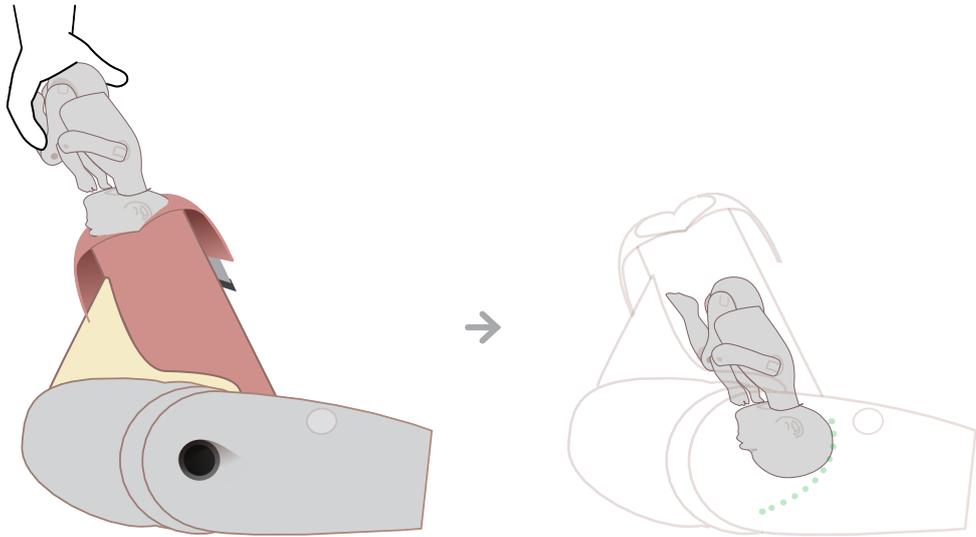






SET UP

SETTING THE MODULE UP FOR VENTOUSE

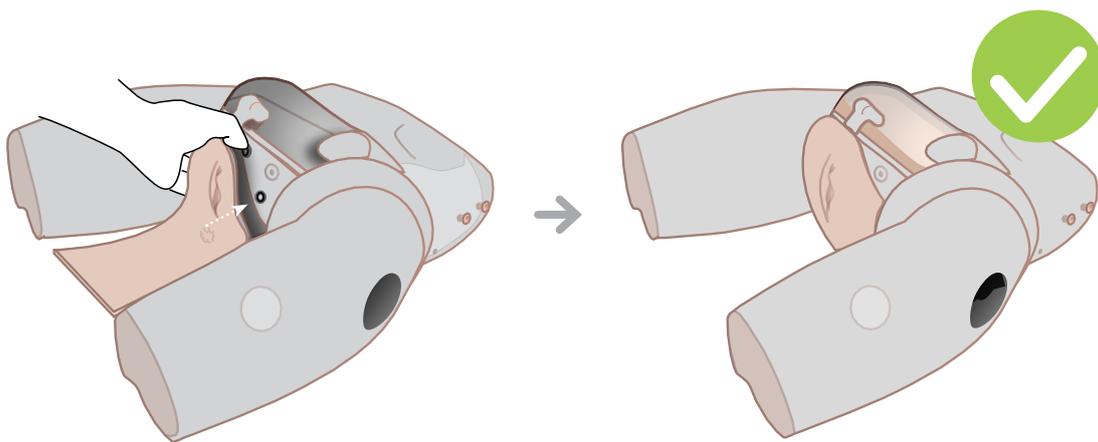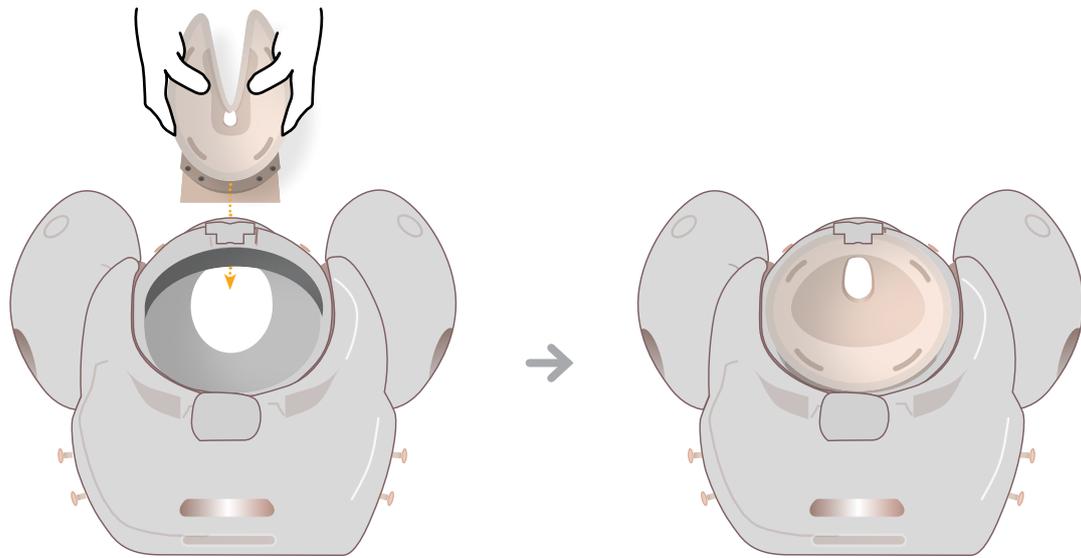





SET UP

SETTING THE MODULE UP FOR VENTOUSE

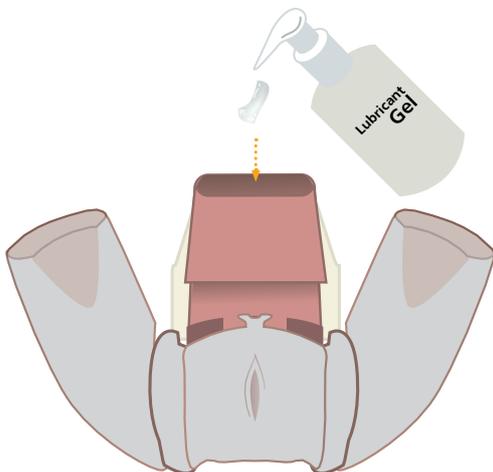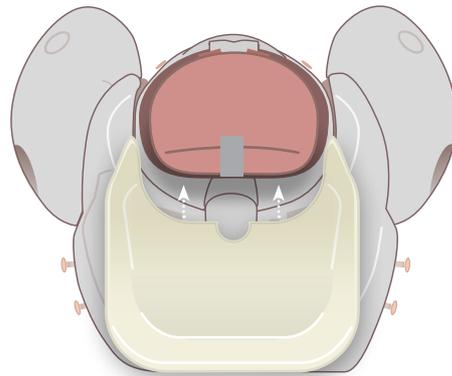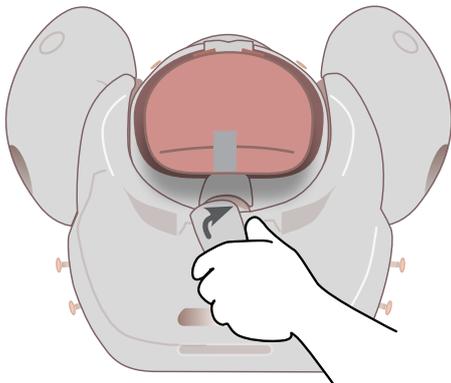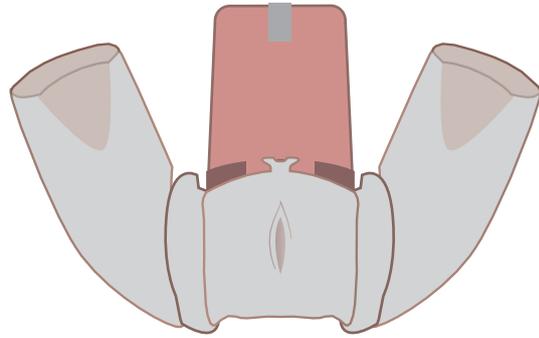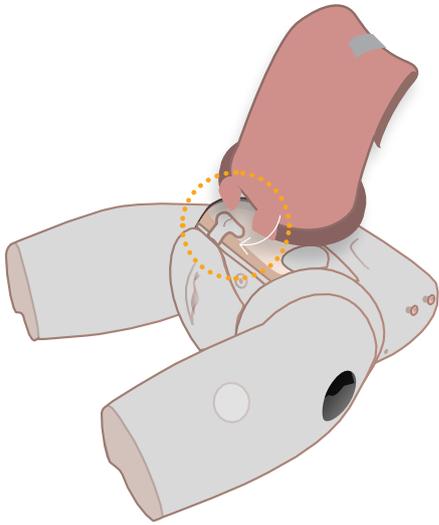






Clean all lubricant off baby head before fitting mask

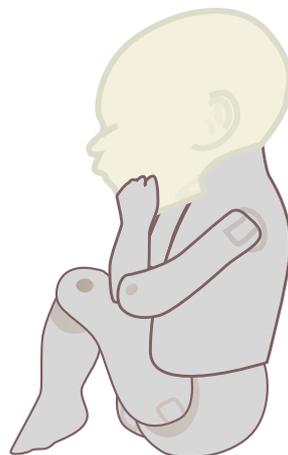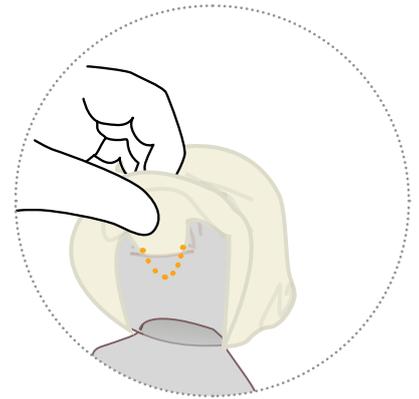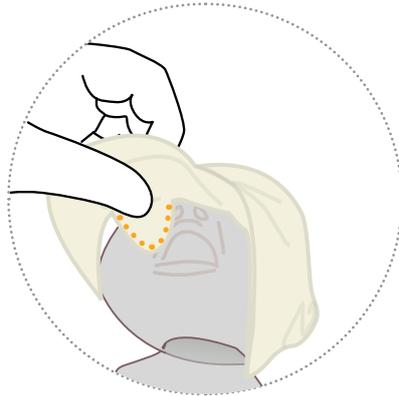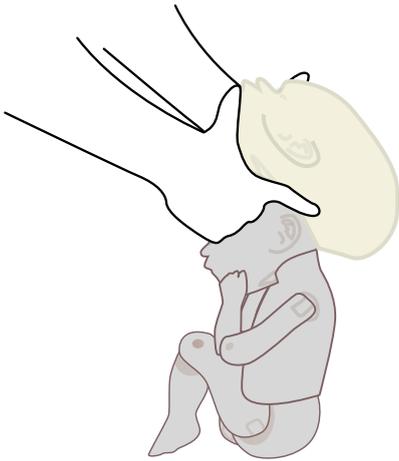

SET UP


SETTING THE MODULE UP FOR VENTOUSE

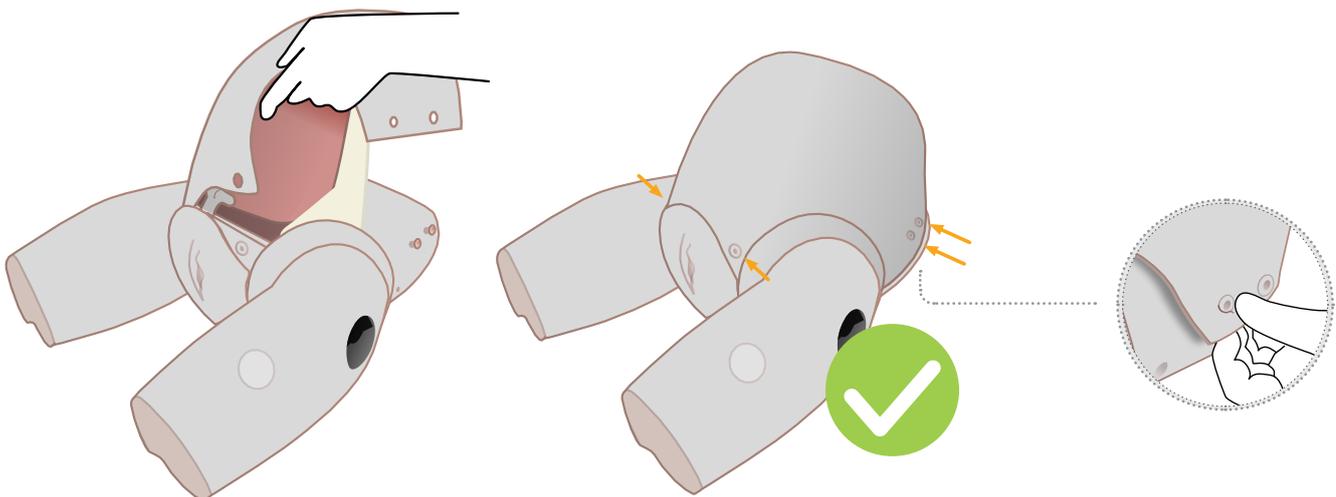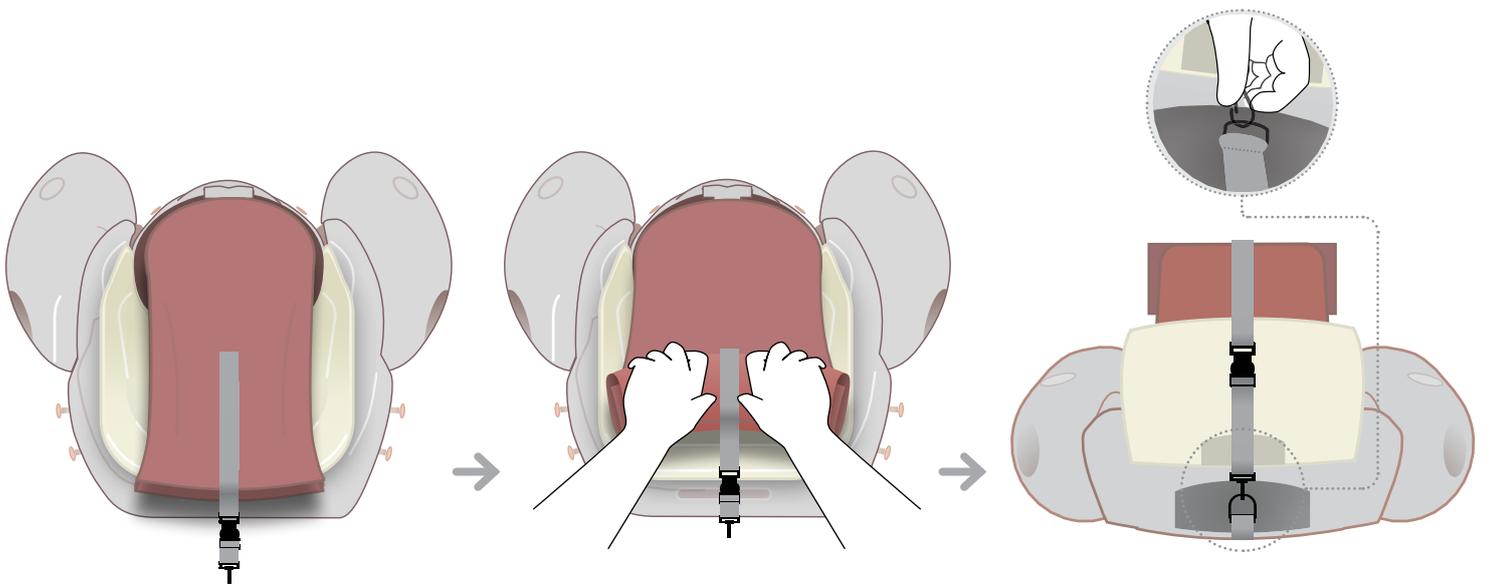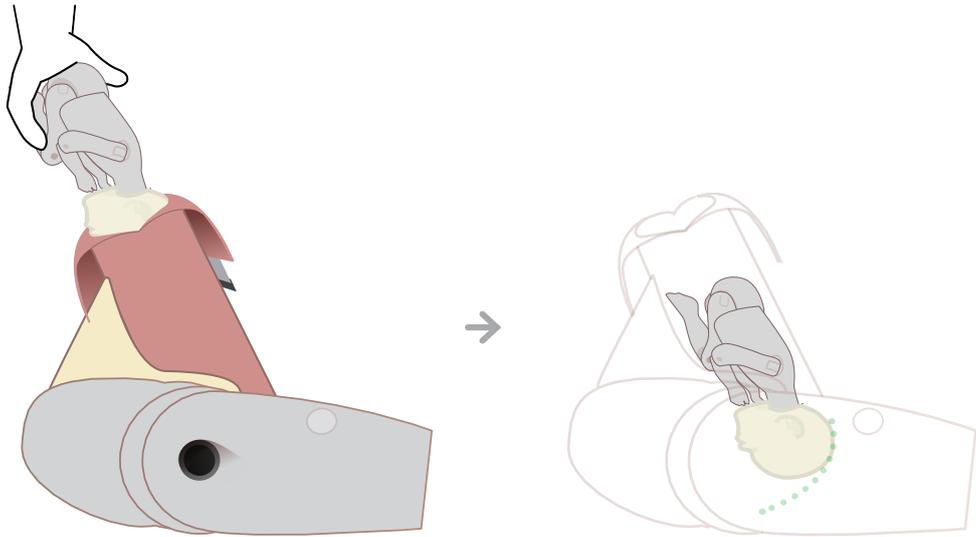

CLEANING & STORAGE

ADDING/REMOVING PERINEUM

After each training session, clean all the lubrication from all mother and baby surfaces with a warm damp cloth.

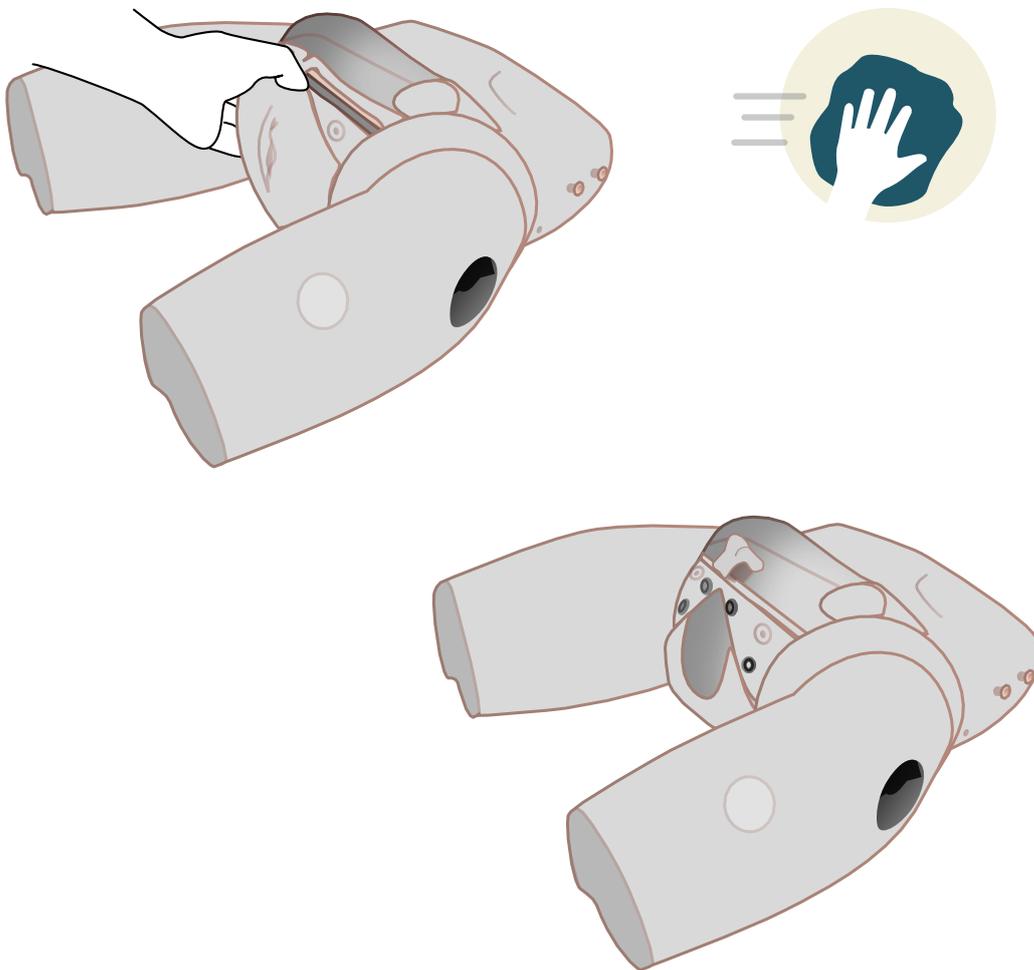

ADDITIONAL MODULES *
ALLOW YOU TO TAILOR
YOUR TRAINING NEEDS

LOWER LEGS -
PROMPT FLEX




CERVICAL
CERCLAGE




CERVICAL
DILATATION
& EFFACEMENT



PPH

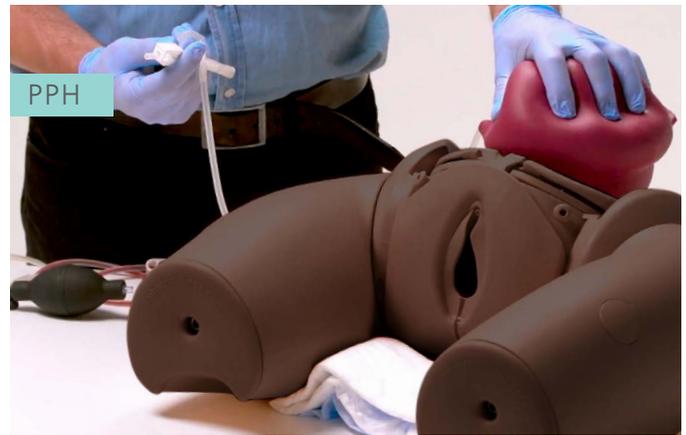

COMPRESSION
SUTURE UTERUS

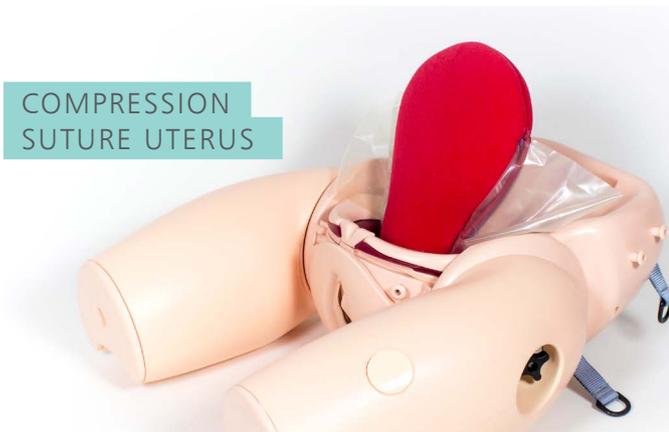

* All available in dark and light skin tones.