

User Guide

PROFESSIONAL ABDOMINAL OPENING & CLOSURE TRAINER

60430



For more skills training products visit limbsandthings.com

Limbs & Things Ltd.
Sussex Street, St Philips,
Bristol, BS2 ORA, UK
E: sales@limbsandthings.com
T: +44 (0) 117 311 0500

PROFESSIONAL ABDOMINAL OPENING & CLOSURE TRAINER

60430

An advanced abdominal wall pad consisting of epidermis, dermis, fat and linea alba. This pad has a separate peritoneal layer and is mounted on a base for teaching abdominal surgical access and closure techniques.

Skills

- Insertion of a veress needle
- Insertion of a trochar
- Hassen technique
- DPL techniques
- Incisions: linear, ellipse, flaps, shaped
- Subcuticular undermining

- Simple and advanced interrupted suturing techniques
- Subcuticular suturing
- Continuous suturing
- Stapling
- Use of adhesive strips

Product Qualities

Anatomy

 5 layered system represents the abdominal wall anatomy: epidermis, dermis, fat, linea alba, peritoneum

Realism

 Abdominal wall is held under tension by a balloon which represents the intestines

Safety

Contains latex

Package supplied

- 50060 BSS Professional AOCT Pad
- 50055 BSS AOCT Frame
- Balloons (x5)
- Base

Handling & care

The Linea Alba of the AOCT Pad and the Peritoneum contain natural latex. Persons allergic to latex should wear latex free gloves when handling the Trainer.

Wash your hands before touching the foam. If the foam is handled after contact with certain metals, such as copper coins, it becomes yellow and discoloured.

Practice normal hygiene after handling the Trainer.

ASSEMBLING THE AOCT



Fold down the legs of the Support Frame and insert them into the white plastic lugs as shown



Push down on the Support Frame to ensure the legs are fully engaged



Inflate the supplied balloon and push underneath the Support Frame. Maximum diameter for the balloon should be 110mm



Turn the AOCT pad upside down and ensure that the white Peritoneum layer is placed in the centre of the pad, as shown



Place the AOCT Pad on the Support Frame with the Peritoneum facing down



Secure the Pad to the Frame by folding over the four flaps



To find out more visit:

limbsandthings.com

