




USER GUIDE

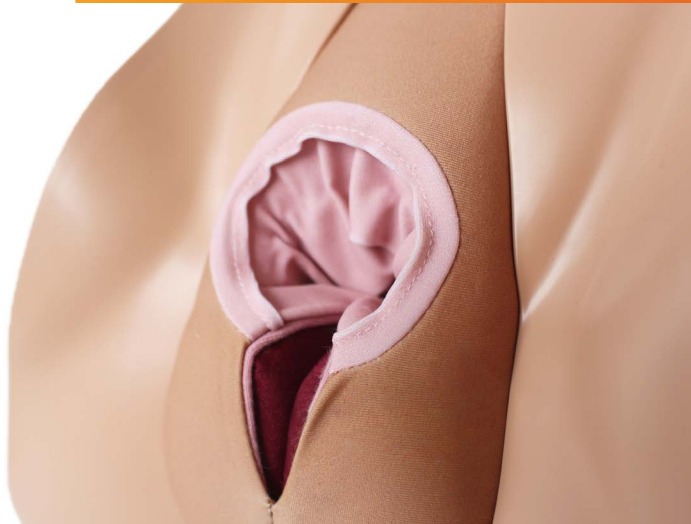

MIDLINE PERINEAL REPAIR TRAINER

This product is available
in light and dark skin tones

61140

61150

FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500


INDEX

PACKAGE SUPPLIED	3
SAFETY & CLEANING INSTRUCTIONS	4
SET UP	5
• SECURING THE STAND	5
• FITTING PERINEAL REPAIR PAD ON TO THE STAND	6
CARE & MAINTENANCE	8

TAKE A LOOK AT OUR **VIDEO**

VISIT THE WEBSITE
limbsandthings.com

OR YOUTUBE CHANNEL
youtube.com/limbsandthings

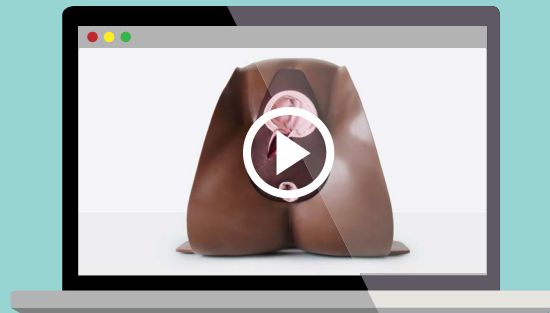

SCAN ME

PACKAGE SUPPLIED

2nd Degree Midline Perineal Repair Pad (x2)

61145

61155



Perineal Repair Stand with Clamps

61109

61129



Perineal Repair Illustration Pack

61112

61132




Perineal Repair Block Loading Handle

61110




YOU'LL NEED (NOT SUPPLIED)

Sutures

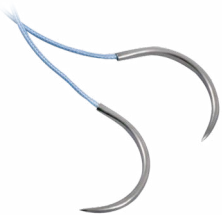

Forceps & Relevant Instruments

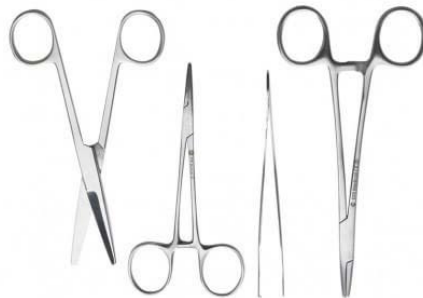

(We recommend using non-tooth forceps only to minimise damage to the repair pad)

SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR PERINEAL REPAIR TRAINER

DO's



Follow all instructions. Keep these instructions. Heed all warnings.

Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.



Handle with care. The silicone is soft and can mark easily.

Remove sutures upon completing a repair. Great care must be taken to ensure that the pad itself isn't damaged. We recommend using small sharp scissors to remove sutures.



DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not use any attachments other than those supplied by the manufacturer.



Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.

Do not allow young children to use this product.




TRANSPORTATION




Always ensure that the product is properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.

1

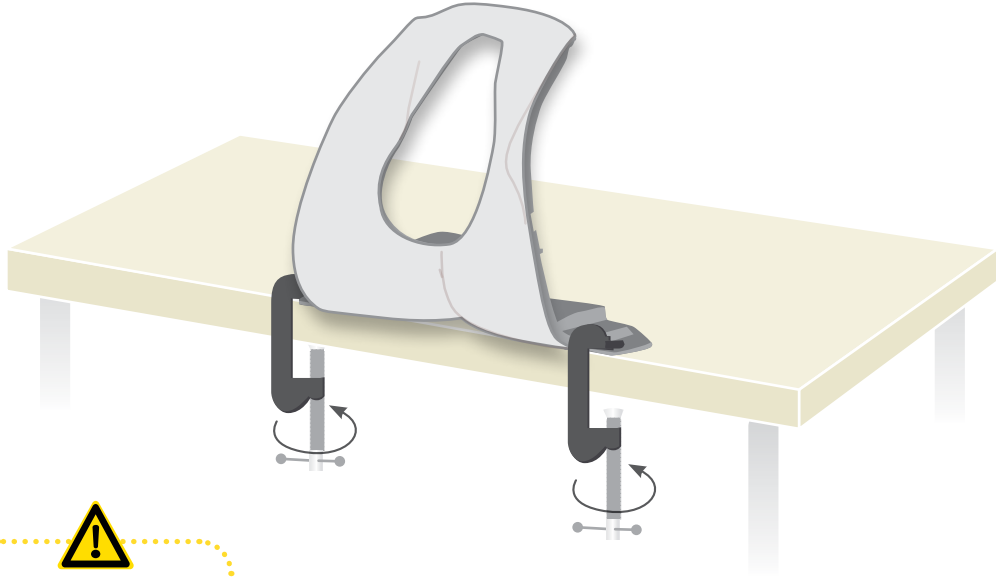

Do not over tighten


2

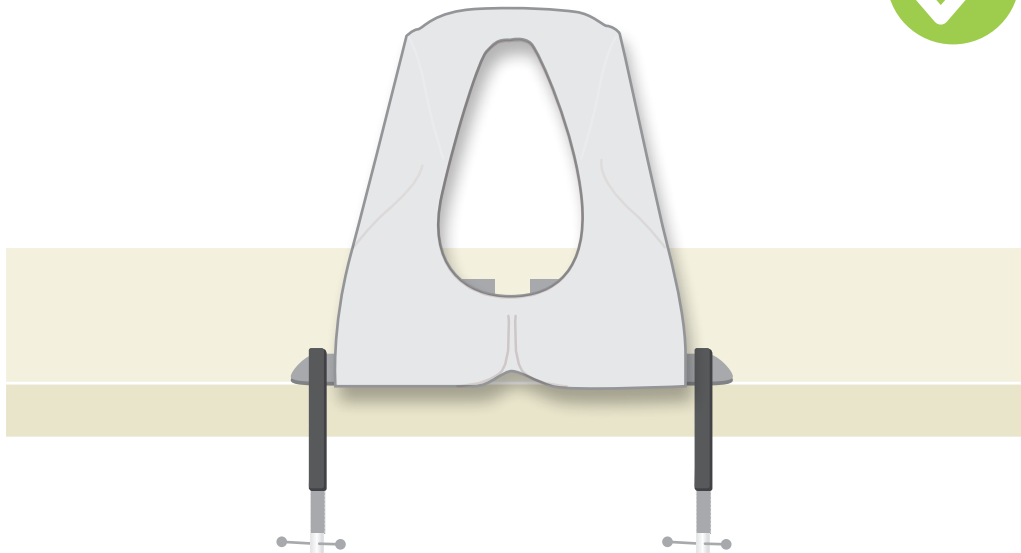

SET UP

FITTING PERINEAL REPAIR PAD ON TO THE STAND


1

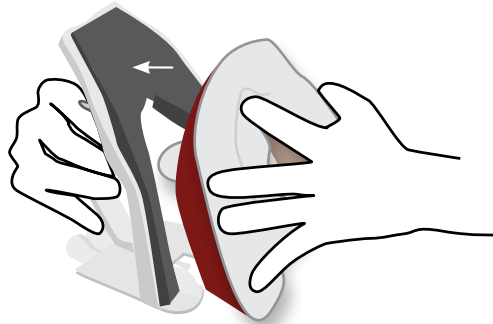

2



3

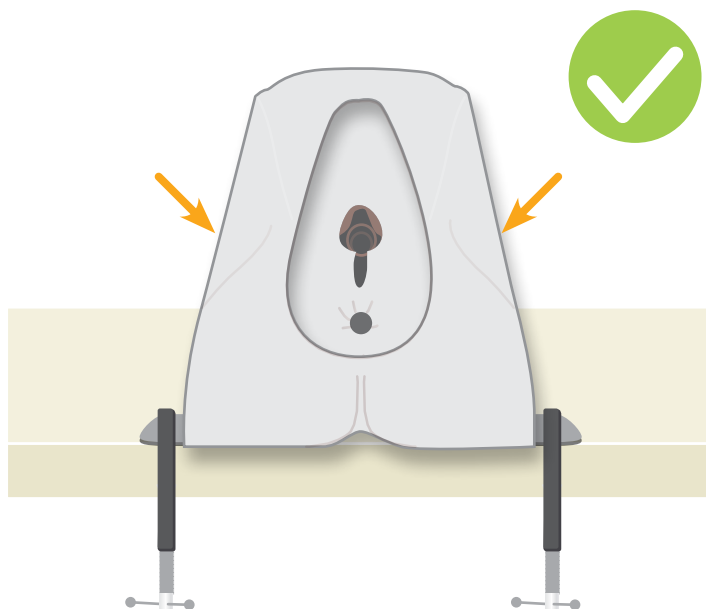

YOU MAY
ALSO BE
INTERESTED IN...

MEDIOLATERAL PERINEAL REPAIR
PAD (x2)

61146

61156



ANAL SPHINCTER
REPAIR BLOCK (x2)

61107

61127



EPISIOTOMY TRAINER MODULE

61104

61124



STAND NOT INCLUDED